



TRAINING UPDATE

**Lab Location:** SGMC & WOMC  
**Department:** Specimen Processing, Core lab

**Date Distributed:** 3/13/25  
**Due Date:** 3/31/25  
**Implementation:** Review for current SOP

DESCRIPTION OF PROCEDURE REVISION

Name of procedure:
SGAH.S31 Autotransfusion (Perfusion) QC Testing
Description of review:
Re-training for Processing and technical staff on Autotransfusion QC testing.

Document your compliance with this training update by taking the quiz in the MTS system.

## SGAH.S31 Autotransfusion (Perfusion) QC Testing

Copy of version 7.0 (approved and current)

Last Approval or  
Periodic Review Completed 5/31/2024

Next Periodic Review  
Needed On or Before 5/31/2026

Effective Date 5/31/2024

Uncontrolled Copy printed on 3/7/2025 4:27 PM

Printed By Demetra Collier (110199)

Organization Adventist HealthCare

### Approval and Periodic Review Signatures

Type	Description	Date	Version	Performed By	Notes
Approval	Lab Director	5/31/2024	7.0	<i>Nicolas Cacciabeve MD</i> Nicolas Cacciabeve	
Approval	Field Ops approval	5/31/2024	7.0	Stephanie Codina	
Periodic review	FO approval	11/13/2023	6.0	Stephanie Codina	
Periodic review	FO approval	11/18/2021	6.0	Stephanie Codina	
Approval	Lab Director	11/15/2019	6.0	Nicolas Cacciabeve	
Approval	Field Ops approval	11/15/2019	6.0	Stephanie Codina	
Approval	QA approval	11/11/2019	6.0	Leslie Barrett	
Periodic review	FO approval	11/1/2019	5.0	Stephanie Codina	
Approval Captured outside MediaLab	Lab Director	10/2/2017	5.0	Nicolas Cacciabeve	Recorded on 1/30/2019 by Leslie Barrett (104977) when document added to Document Control
Periodic review Captured outside MediaLab	Designated Reviewer	10/2/2017	5.0	Nicolas Cacciabeve	Recorded on 1/30/2019 by Leslie Barrett (104977) when document added to Document Control

Approvals and periodic reviews that occurred before this document was added to Document Control may not be listed.

### Version History

Version	Status	Type	Date Added	Date Effective	Date Retired
7.0	Approved and Current	Major revision	5/31/2024	5/31/2024	Indefinite
6.0	Retired	Major revision	11/11/2019	12/2/2019	5/31/2024
5.0	Retired	First version in Document Control	1/30/2019	10/17/2017	12/2/2019

### Linked Documents

- AG.F267 Autotransfusion QC Requisition, WOMC



Non-Technical SOP

<b>Title</b>	<b>Autotransfusion (Perfusion) QC Testing</b>	
<b>Prepared by</b>	Marie Sabonis	Date: 1/26/2010
<b>Owner</b>	Stephanie Codina	Date: 9/22/2017

<b>Laboratory Approval</b>		
<b>Print Name and Title</b>	<b>Signature</b>	<b>Date</b>
<i>Refer to the electronic signature page for approval and approval dates.</i>		
Local Issue Date:		Local Effective Date:

**TABLE OF CONTENTS**

1.	PURPOSE.....	1
2.	SCOPE.....	1
3.	RESPONSIBILITY .....	1
4.	DEFINITIONS .....	1
5.	PROCEDURE.....	2
6.	RELATED DOCUMENTS .....	3
7.	REFERENCES .....	3
8.	REVISION HISTORY .....	3
9.	ADDENDA AND APPENDICES.....	4

**1. PURPOSE**

This procedure describes the process to place laboratory orders and enter results for QC testing for autotransfusion machines.

**2. SCOPE**

This procedure applies to quality control testing of the autotransfusion machines.


**3. RESPONSIBILITY**

This procedure is performed by Specimen Processing and Core Lab staff members.

**4. DEFINITIONS**

N/A

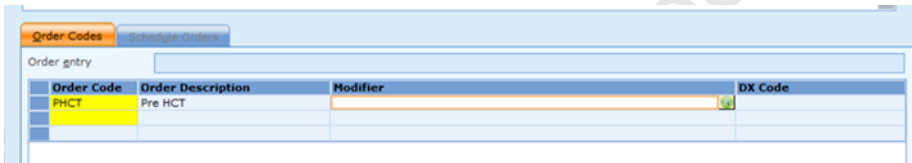

Step	Action
7	<p>Enter the following information:</p> <ul style="list-style-type: none"> <li>A. Collect date</li> <li>B. Collect time</li> <li>C. Received date</li> <li>D. Received time</li> <li>E. Ordering physician <ul style="list-style-type: none"> <li>a. At SGM, type "O12905" for SpecialtyCare, QC</li> <li>b. At WOH, type "O10870" for WOMC, Perfusionist</li> </ul> </li> <li>F. Order account (type 999999 if it does not default).</li> <li>G. Phlebotomist code <ul style="list-style-type: none"> <li>a. At SGM, type "907" for SpecialtyCare QC</li> <li>b. At WOMC, type "906" for WOMC Perfusionist</li> </ul> </li> </ul>
8	<p>At the "Order Code" prompt, type the mnemonic(s) for the test(s) requested on the requisition (note pre- and post-processing labs have different mnemonics). Click the "save" button when all tests have been entered.</p> 
9	<p>Autotransfusion quality control specimens are always run, even if grossly hemolyzed. Results are manually entered into Sunquest.</p>

## 6. RELATED DOCUMENTS

SpecialtyCare Autotransfusion QC, SGM (AG.F696)

Autotransfusion QC Requisition, WOMC (AG.F267)

## 7. REFERENCES

N/A

## 8. REVISION HISTORY

Version	Date	Reason for Revision	Revised By	Approved By
000	6/18/2012	Sections 1,2,5 & 9: Update company name from Biotronics to Procirca	L. Barrett	S. Khandagale
001	9/20/2012	Section 5: Add new cell saver BIOT-5	L. Barrett	S. Khandagale

Adventist HealthCare

Site: Shady Grove Medical Center, White Oak Medical Center

Title: Autotransfusion (Perfusion) QC Testing

002	12/1/2014	Title, Section 1 & 2: remove company name and replace with Autotransfusion or perfusionist Section 4: add GUI Section 5: add item D Section 6: moved form from section 9 Section 9: add SQ 7.1 ordering process Footer: version # leading zero's dropped due to new EDCS in use as of 10/7/13.	L. Barrett S. Khandagale	S. Khandagale
3	6/9/2015	Section 3: add Core Lab Section 4: add autotransfusion and perfusionist Section 5: add item E	L. Barrett	S. Khandagale R SanLuis
4	9.22.2017	Update owner Section 5: Combined SGMC and WAH procedures into a single procedure. Removed SmarTerm ordering instructions. Updated formatting and wording for clarity. Section 6: Added WAH requisition	SCodina	NCacciabeve
5	11.11.19	Header: Changed WAH to WOMC Section 5 & 6: Updated to WOMC perfusion	LBarrett	NCacciabeve
6	5.30.24	Changed SGMC provider and references from Biotronics to SpecialtyCare	SCodina	NCacciabeve

**9. ADDENDA AND APPENDICES**

NA