



TRAINING UPDATE


Lab Location: GEC, SGAH & WAH
Department: Micro

Date Distributed: 8/28/2012
Due Date: 9/30/2012

DESCRIPTION OF PROCEDURE REVISION

| |
|---|
| Name of procedure: |
| Microbiology Gram Stainer PM Chart Wescor Model 7320 form AG.F27.002 |
| Description of change(s): |
| Form revised to match Gram Stain SOP for Clean Cycle frequency (section 8.4) Added columns to document clean cycle performance by Evening and Night shifts |

Document your compliance with this training update by taking the quiz in the MTS system.

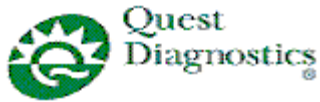

**Microbiology Gram Stainer PM Chart
Wescor Model 7320**

- Germantown Emergency Center
- Shady Grove Adventist Hospital
- Washington Adventist Hospital

Month / Year _____

| Day | Daily: Clean Cycle, Wiping, Daily Control Slides | Clean Cycle | | Nozzle Performance | | | | | Clean Reagent Nozzles and Instrument | | | | | |
|-----|--|-------------|---------------|--------------------|-------------------------------|--------------------------|----------------|----------------------------------|--------------------------------------|-----------------|------------------------------|-------------------------------|---|------|
| | | Eve | Night | Week | Reagent (Acceptable Range-mL) | Spray Pattern √ if OK | Spray Volume | Record New Vol if Nozzle Cleaned | Tech | Reagent | Previous Month Ending Volume | Volume after Monthly Cleaning | Disinfect DI Water bottle √ if cleaned | Tech |
| 1 | | | | 1 | A (10-12) | | | | | A | | | N/A | |
| 2 | | | B (9-11) | | | | | | | | | | | |
| 3 | | | C (9-11) | | | | | | | | | | | |
| 4 | | | | 2 | D Front (9-11) | | | | | B | | | N/A | |
| 5 | | | D Back (9-11) | | | | | | | | | | | |
| 6 | | | E (9-11) | | | | | | | | | | | |
| 7 | | | | 3 | A (10-12) | | | | | C | | | N/A | |
| 8 | | | B (9-11) | | | | | | | | | | | |
| 9 | | | C (9-11) | | | | | | | | | | | |
| 10 | | | | 4 | D Front (9-11) | | | | | D Front | | | N/A | |
| 11 | | | D Back (9-11) | | | | | | | | | | | |
| 12 | | | E (9-11) | | | | | | | | | | | |
| 13 | | | | 1 | A (10-12) | | | | | D Back | | | N/A | |
| 14 | | | B (9-11) | | | | | | | | | | | |
| 15 | | | C (9-11) | | | | | | | | | | | |
| 16 | | | | 2 | D Front (9-11) | | | | | E | | | N/A | |
| 17 | | | D Back (9-11) | | | | | | | | | | | |
| 18 | | | E (9-11) | | | | | | | | | | | |
| 19 | | | | 3 | A (10-12) | | | | | | | | N/A | |
| 20 | | | B (9-11) | | | | | | | | | | | |
| 21 | | | C (9-11) | | | | | | | | | | | |
| 22 | | | | 4 | D Front (9-11) | | | | | | | | N/A | |
| 23 | | | D Back (9-11) | | | | | | | | | | | |
| 24 | | | E (9-11) | | | | | | | | | | | |
| 25 | | | | 1 | A (10-12) | | | | | | | | N/A | |
| 26 | | | B (9-11) | | | | | | | | | | | |
| 27 | | | C (9-11) | | | | | | | | | | | |
| 28 | | | | 2 | D Front (9-11) | | | | | | | | N/A | |
| 29 | | | D Back (9-11) | | | | | | | | | | | |
| 30 | | | E (9-11) | | | | | | | | | | | |
| 31 | | | | 3 | A (10-12) | | | | | | | | N/A | |
| | | | B (9-11) | | | | | | | | | | | |
| | | | C (9-11) | | | | | | | | | | | |
| | | | | 4 | D Front (9-11) | | | | | | | | N/A | |
| | | | D Back (9-11) | | | | | | | | | | | |
| | | | E (9-11) | | | | | | | | | | | |
| | | | | Weekly review: | | | Weekly review: | | | Weekly review: | | | | |
| | | | | Weekly review: | | | Weekly review: | | | Monthly review: | | | | |

Instructions for Spray Pattern test, Spray Volume test, and cleaning nozzles are contained in the User's Manual.