

## DXC 800 (GEN) GENTAMICIN

- |   |  |  |
|---|--|--|
| <input checked="" type="checkbox"/> St. Joseph Medical Center, Tacoma, WA | <input type="checkbox"/> St. Anthony Hospital Gig Harbor, WA | <input type="checkbox"/> Harrison Medical Center, Bremerton, WA  |
| <input type="checkbox"/> St. Francis Hospital, Federal Way, WA            | <input type="checkbox"/> St. Elizabeth Hospital Enumclaw, WA | <input type="checkbox"/> Harrison Medical Center, Silverdale, WA |
| <input type="checkbox"/> St. Clare Hospital Lakewood, WA                  | <input type="checkbox"/> Highline Medical Center Burien, WA  | <input type="checkbox"/> PSC                                     |

### PURPOSE

To provide instructions for the quantitative determination of gentamicin on the DXC 600/800.

### PRINCIPLE

GEN reagent, when used in conjunction with UniCel® DxC 600/800 System(s) and SYNCHRON® Systems Drug Calibrator 3 Plus, is intended for quantitative determination of Gentamicin concentration in human serum or plasma.

### BACKGROUND

#### Clinical Significance

Gentamicin is an antibiotic used to treat serious gram-negative bacterial infections. Gentamicin therapy is monitored for effectiveness of the dose and possible nephrotoxicity.

#### Methodology

GEN reagent is used to measure the GEN concentration by a particle enhanced turbidimetric inhibition immunoassay method. A particle-bound drug (PBD) binds to GEN specific antibody (Ab) resulting in the formation of insoluble aggregates causing light scatter. Non-particle-bound GEN in the patient sample competes with the PBD for the antibody binding sites, inhibiting formation of insoluble aggregates. The rate and amount of particle aggregation is inversely proportional to the concentration of GEN in the sample. The SYNCHRON® System(s) automatically proportions the appropriate sample and reagent volumes into a cuvette. The ratio used is one part sample to 105 parts reagent. The system monitors the aggregate formation by measuring the change in absorbance at 380 nanometers. This change in absorbance is inversely proportional to the concentration of gentamicin in the sample and is used by the System to calculate and express the gentamicin concentration based upon a multi-point calibration curve.



E015222L.EPS

### RELATED DOCUMENTS

- |              |  |
|--------------|--|
| R-PO-CH-0810 | Quality Control Program General Laboratory |
| R-PO-CH-0809 | Quality Control Westgard Rules Statistics  |
| R-PR-AD-0540 | Specimen Rejection/Cancellation Protocol   |
| J-F-CH-0820  | DXC 800 Controls                           |
| J-F-CH-0826  | DXC 800 Calibrators                        |
| J-F-CH-1940  | DXC 800 (AMR) Analytical Measurement Range |

## SPECIMEN

### Type of Specimen

Biological fluid samples should be collected in the same manner routinely used for any laboratory test. Freshly drawn serum or plasma are the preferred specimens. Acceptable anticoagulants are listed in the PROCEDURAL NOTES section of this chemistry information sheet. Whole blood or urine are not recommended for use as a sample.

### Specimen Storage and Stability

1. Tubes of blood are to be kept closed at all times and in a vertical position. It is recommended that the serum or plasma be physically separated from contact with cells within two hours from the time of collection.
2. Separated serum or plasma should not remain at room temperature longer than 8 hours. If assays are not completed within 8 hours, serum or plasma should be stored at +2°C to +8°C. If assays are not completed within 48 hours, or the separated sample is to be stored beyond 48 hours, samples should be frozen at -15°C to -20°C. Frozen samples should be thawed only once. Analyte deterioration may occur in samples that are repeatedly frozen and thawed.

Sample Type	Volume	Sample Stability
Plasma/Serum	0.5mL	<ul style="list-style-type: none"><li>• Separate serum from cells within 2 hours</li><li>• Room Temp 8 hours</li><li>• Refrigerated 48 hours</li><li>• Frozen 3 months</li></ul>

### Criteria for Unacceptable Specimens

See Specimen Rejection/Cancellation Protocol

### Sample Volume

A filled 0.5 mL sample cup is the optimum volume. For optimum primary sample tube volumes in primary tube samples and minimum volumes, refer to the Primary Tube Sample Template for your system.

## REAGENTS

### Contents

Each kit contains the following items:  
Two GEN Reagent Cartridges (2 x 100 tests)

Volume per Test	
Sample Volume	3 µL
Total Reagent Volume	315 µL
Cartridge Volumes	A 245 µL B 40 µL C 30 µL

Reactive Ingredients	
Gentamicin Particle Reagent	6.8 mL
Monoclonal anti-Gentamicin Antibodies (mouse)	4.7 mL
Gentamicin Reaction Buffer	110.0 mL



### CAUTION

Sodium azide preservative may form explosive compounds in metal drain lines. See National Institute for Occupational Safety and Health Bulletin: Explosive Azide Hazards (8/16/76).

Also non-reactive chemicals necessary for optimal system performance.

### Reagent Preparation

No preparation is required. Do not mix.

### Acceptable Reagent Performance

The acceptability of a reagent is determined by successful calibration and by ensuring that quality control results are within your facility's acceptance criteria.

### Reagent Storage and Stability

GEN reagent when stored unopened at +2°C to +8°C, will remain stable until the expiration date printed on the cartridge label. Once opened, the reagent is stable for 42 days at +2°C to +8°C unless the expiration date is exceeded. DO NOT FREEZE. Do not expose reagent to temperatures above +35°C or to direct sunlight.

## CALIBRATION

### Calibrator Required

SYNCHRON<sup>®</sup> Systems Drug Calibrator 3 Plus

### Calibrator Preparation

No preparation is required.

### Calibrator Storage and Stability

SYNCHRON<sup>®</sup> Systems Drug Calibrator 3 Plus is stable until the expiration date printed on the calibrator bottle if stored capped in the original container at +2°C to +8°C.

### Calibration Information

1. The system must have a valid calibration curve in memory before control or patient samples can be run.
2. Under typical operating conditions the GEN reagent cartridge must be calibrated every 14 days and also with certain parts replacements or maintenance procedures, as defined in the UniCel DxC 600/800 System *Instructions For Use* (IFU) manual. This assay has within-lot calibration available. Refer to the UniCel DxC 600/800 System *Instructions For Use* (IFU) manual for information on this feature.

3. For detailed calibration instructions, refer to the UniCel DxC 600/800 System *Instructions For Use* (IFU) manual.
4. The system will automatically perform checks on the calibration and produce data at the end of calibration. In the event of a failed calibration, the data will be printed with error codes and the system will alert the operator of the failure. For information on error codes, refer to the UniCel DxC 600/800 System *Instructions For Use* (IFU) manual.

### Traceability

For Traceability information refer to the Calibrator instructions for use.

### QUALITY CONTROL

See Related Documents J-F-CH0820 DXC 800 controls

### STEPS

1. If necessary, load the reagent onto the system.
2. After reagent load is completed, calibration may be required.
3. Program samples and controls for analysis.
4. After loading samples and controls onto the system, follow the protocols for system operation. To load samples manually refer to the FHS DXC Series Manual Sample Programming procedure. For detailed testing procedures, refer to the UniCel DxC 600/800 System *Instructions For Use* (IFU) manual.

For detailed testing procedures, refer to the UniCel DxC 600/800 System *Instructions For Use* (IFU) manual.

### CALCULATIONS

SYNCHRON<sup>®</sup> System(s) perform all calculations internally to produce the final reported result. The system will calculate the final result for sample dilutions made by the operator when the dilution factor is entered into the system during sample programming.

### ANTICOAGULANT TEST RESULTS

If plasma is the sample of choice, the following anticoagulants were found to be compatible with this method based on a study of 20 healthy volunteers:

Anticoagulant	Level Tested for In Vitro Interference
Lithium Heparin	14 Units/mL
Sodium Heparin	14 Units/mL

### PERFORMANCE CHARACTERISTICS

#### Reference Range

Sample Type	Male and Female		Critical High
	Age	Range	
Trough	0-4 weeks	0.5-1.0 µg/mL	>1.0 µg/mL
Trough	>4 weeks	0.0-2.0 µg/mL	>2.0 µg/mL
Peak	All	5.0-10.0 µg/mL	>10.0 µg/mL
Random	All	N/A	N/A

For Critical Value reporting protocol, refer to FHS Critical Policy

### Analytic Range

The SYNCHRON® System(s) method for the determination of this analyte provides the following analytical ranges:

Sample Type	Conventional Units
Serum or Plasma	0.5 – 12.0 µg/mL

Samples with concentrations outside of the analytical range will be reported as "<0.5 µg/mL" ("<1.0 µmol/L") . Samples with activities greater than the analytical range may be confirmed by diluting with saline and reanalyzing. The appropriate dilution factor should be applied to the reported result.

The analytical range of this assay is 0.5-12.0 µg/mL. Very rarely, a patient sample may contain a nonspecific protein which could cause a false low GEN result. It is recommended that the low limit of the reportable range of this assay be set to the default value of 0.1 µg/mL. All samples with printed results below 0.1 µg/mL will need to be confirmed by dilution. Printed results between 0.1 µg/mL and 0.49 µg/mL do not need to be confirmed by dilution and can be reported as "<0.5 µg/mL".

Dilution protocol: Confirm a suspected low GEN sample result by adding one measured volume of test sample to an equal volume of Multiquel control level 1. The assayed GEN result of this diluted sample should be approximately half of the value of the Multiquel control level 1. The confirmed result should be reported out as "<0.5 µg/mL".

If the assayed result of the diluted sample is not close to half of the Multiquel control level 1 value, send specimen to PAML for testing.

### Reporting results outside of analytical range

Lower limit of detection	0.5 µg/mL	Results below 0.5, report as <0.5 µg/mL  Results below 0.1, dilute 1:2 with Multiquel level 1 and the diluted result should be approximately half of the value of the Multiquel level 1 values, if not send to PAML. The confirmed result should be reported as <0.5 µg/mL
Upper limit of detection	12.0 µg/mL	Results >12.0, dilute with 0.9% saline, reanalyze and dilution factor applied. The maximum allowable dilution is X2. Results >24.0 are reported as >24.0 µg/mL

### Sensitivity

Sensitivity is defined as the lowest measurable concentration which can be distinguished from zero with 95% confidence. Sensitivity for GEN determination is 0.5 µg/mL (1.0 µmol/L).

### LIMITATIONS

None identified.

## Interferences

1. The following substances were tested for interference with this methodology:

Substance	Source	Level Tested	Observed Effect
Hemoglobin	RBC hemolysate	500 mg/dL INDEX of 10	No significant Interference (within $\pm$ 0.4 $\mu$ g/mL or 10%)
Bilirubin	Porcine	30 mg/dL INDEX of 20	No significant Interference (within $\pm$ 0.4 $\mu$ g/mL or 10%)
Rheumatoid Factor	Human	300 IU/mL	No significant Interference (within $\pm$ 0.4 $\mu$ g/mL or 10%)
Lipemia	Human	4+ INDEX of 10	No significant Interference (within $\pm$ 0.4 $\mu$ g/mL or 10%)
Paraprotein (IgM)	Human	500 mg/dL	No significant Interference (within $\pm$ 0.4 $\mu$ g/mL or 10%)

2. Refer to References for other interferences caused by drugs, disease and preanalytical variables.

3. For assays employing mouse antibodies, the possibility exists for interference by human anti-mouse antibodies (HAMA) in the sample. Human anti-mouse antibodies may be present in samples from patients who have received immunotherapy or diagnostic procedures utilizing monoclonal antibodies or in individuals who have been regularly exposed to animals. Additionally, other heterophile antibodies, such as human anti-goat antibodies may be present in patient samples. Interpretation of results should be done in the context of the overall clinical presentation of the patient, including symptoms, clinical history, data from additional tests and other appropriate information.


## ADDITIONAL INFORMATION

For more detailed information on UniCel DxC Systems, refer to the appropriate system manual.

## REFERENCES

1. Newman, D. J., Henneberry, H., Price, C. P., "Particle Enhanced Light Scattering Immunoassay", *Ann. Clin. Biochem.*, 29:22-42 (1992).
2. Tietz, N. W., "Specimen Collection and Processing; Sources of Biological Variation", *Textbook of Clinical Chemistry*, 2nd Edition, W. B. Saunders, Philadelphia, PA (1994).
3. National Committee for Clinical Laboratory Standards, *Procedures for the Handling and Processing of Blood Specimens*, Approved Guideline, NCCLS publication H18-A, Villanova, PA (1990).
4. CDC-NIH manual, *Biosafety in Microbiological and Biomedical Laboratories*, U.S. Government Printing Office, Washington, D.C. (1984).
5. Burtis, C. A., Ashwood, E. R., *Tietz Textbook of Clinical Chemistry*, 2nd Edition, W. B. Saunders, Philadelphia, PA (1994).
6. National Committee for Clinical Laboratory Standards, *How to Define, Determine, and Utilize Reference Intervals in the Clinical Laboratory*, Approved Guideline, NCCLS publication C28-A, Villanova, PA (1995).
7. Tietz, N. W., ed., *Fundamentals of Clinical Chemistry*, 3rd Edition, W. B. Saunders, Philadelphia, PA (1987).
8. Henry, J. B., *Clinical Diagnosis and Management by Laboratory Methods*, 18th Edition, W. B. Saunders Company, Philadelphia, PA (1991).
9. Young, D. S., *Effects of Drugs on Clinical Laboratory Tests*, 4th Edition, AACC Press, Washington, D. C. (1995).
10. Friedman, R. B., Young, D. S., *Effects of Disease on Clinical Laboratory Tests*, 3rd Edition, AACC Press, Washington, D.C. (1997).
11. Young, D. S., *Effects of Preanalytical Variables on Clinical Laboratory Tests*, 2nd Edition, AACC Press, Washington, D. C. (1997).

12. Bjerner, J., et al., "Immunometric Assay Interference: Incidence and Prevention", *Clin. Chem.* 48:613 621 (2002).
13. Kricka, L. J., "Interferences in Immunoassays-Still a Threat", *Clin. Chem.*, 46:1037 1038 (2000).
14. National Committee for Clinical Laboratory Standards, *Method Comparison and Bias Estimation Using Patient Samples*, Approved Guideline, NCCLS publication EP9-A, Villanova, PA (1995).
15. National Committee for Clinical Laboratory Standards, *Precision Performance of Clinical Chemistry Devices*, Tentative Guideline, 2nd Edition, NCCLS publication EP5-T2, Villanova, PA (1992).

<b>DOCUMENT APPROVAL Purpose of Document / Reason for Change:</b>			
Formatting, added related documents, added max dilutions, changed ref ranges to match LIS.			
<input type="checkbox"/> No significant change to process in above revision. Per CAP, this revision does not require further Medical Director approval.			
<b>Committee Approval Date</b>	<input type="checkbox"/> Date: <input checked="" type="checkbox"/> N/A – revision of department-specific document which is used at only one facility	<b>Medical Director Approval</b> (Electronic Signature)	 9/25/15