

ACL TOP - ADDING A TEST

- | | | |
|--|---|---|
| <input checked="" type="checkbox"/> St. Joseph Medical Center Tacoma, WA | <input checked="" type="checkbox"/> St. Clare Hospital Lakewood, WA | <input checked="" type="checkbox"/> St. Elizabeth Hospital Enumclaw, WA |
| <input checked="" type="checkbox"/> St. Francis Hospital Federal Way, WA | <input checked="" type="checkbox"/> St. Anthony Hospital Gig Harbor, WA | <input type="checkbox"/> PSC |

PURPOSE

To provide instructions for adding a test not ordered through the Laboratory Information System (LIS) on the ACL TOP Family Analyzer.

BACKGROUND

Samples that are already loaded onto the analyzer that have test(s) added through the LIS on to the same accession number will automatically begin to run if the Auto-Run feature is enabled. However, there are times tests may need to be manually added.

RELATED DOCUMENTS

- | | |
|-------------|--|
| R-W-CG-2090 | ACL TOP - Sample Analysis-Barcoded Specimens |
| R-W-CG-2091 | ACL TOP - Sample Analysis-Non-Barcoded Specimens |
| R-W-CG-1131 | ACL TOP - Reagent Management-Assay Reagents |
| R-W-CG-1132 | ACL TOP - Reagent Management-Bulk Reagents |

PROCEDURE STEPS

1. Access the sample rack detail screen of the rack where the sample is located.
2. Double click on the empty test request field of the sample to open the tests and profiles window.
3. Choose the test(s) to be added to the sample
4. The test will run automatically if the analyzer is running. If not, touch the run button to start analysis.

Or

1. Access the sample list.
2. Select and place a check by the desired sample
3. Single click or touch the add remove test button from the tool bar.
4. Choose the test(s) to be added to the sample
5. The test will run automatically if the analyzer is running. If not touch the run button to start analysis.

REFERENCES

- ACL TOP Family On-Line Help Manual
- ACL TOP Family Training Manual