

VIDAS 3 MAINTENANCE LOG

- | | | |
|---|---|--|
| <input type="checkbox"/> St. Joseph Medical Center Tacoma, WA | <input type="checkbox"/> St. Clare Hospital Lakewood, WA | <input type="checkbox"/> St. Elizabeth Hospital Enumclaw, WA |
| <input type="checkbox"/> St. Francis Hospital Federal Way, WA | <input checked="" type="checkbox"/> St. Anthony Hospital Gig Harbor, WA | <input checked="" type="checkbox"/> Highline Medical Center Burien, WA |
| | | <input type="checkbox"/> PSC |

Month: _____ Year: _____

Daily Maintenance	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Record Temp: Component A SPR Block																															
Record Temp: Component A Tray Temp																															
Record Temp: Component B SPR Block																															
Record Temp: Component B Tray Temp																															
Record Temp: Component C SPR Block																															
Record Temp: Component C Tray Temp																															
Record Temp: Component D SPR Block																															
Record Temp: Component D Tray Temp																															
Record Temp: Cooling Unit																															
Record Temp: Internal Instrument Temp																															
QC performed, printed and recorded in LIS																															
Tech ID																															
Monthly Maintenance (15 th of month)	Date/Tech ID																6-month Maintenance (Apr & Oct)		Date/Tech ID												
*** Clean SPR Block																	*** Clean housing, front cover & touch screen														
*** Clean Reagent Strip Sections																	*** Clean vials, tubes & disposables rack														
Tech ID																	*** Clean waste drawer														

***Turn VIDAS 3 off prior to performing these tasks

Reviewed by/date: _____