



State of California

Department of Health Services

Prevention Services

Division of Communicable Disease Control

Microbial Diseases Laboratory Branch

David Jensen, Public Health Microbiologist II

Botulinum neurotoxin producing species of *Clostridium*

- *Clostridium botulinum*
- *C. baratii*
- *C butyricum*

Botulism

IF BOTULISM POISONING IS SUSPECTED—

NOTIFY YOUR

**STATE PUBLIC HEALTH LABORATORY
OR**

**LOS ANGELES COUNTY PUBLIC HEALTH
LABORATORY
(IF IN LOS ANGELES)**


IMMEDIATELY FOR EVALUATION AND REFERRAL

Botulism Types

**Intentional
Contamination**



Foodborne

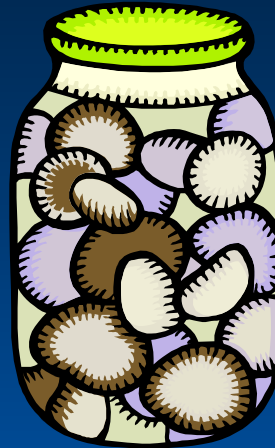

**Infant
(& adult colonization)**



Wound




Botulinum toxin types

- **Seven known toxins:A,B,C,D,E,F,G**
 - **Human botulism**
 - **A, B, E, F (C & D were described in late 1950's)**
 - **Animal**
 - **B, C, D, E**
 - **Natural disease unidentified**
 - **Type G**

Botulism


- **The preliminary diagnosis of botulism is made clinically, i.e., based on the patient's case history and physical finding**
- **Botulinum toxin, or botox, produces a neuromuscular illness**

Botulinum neurotoxin

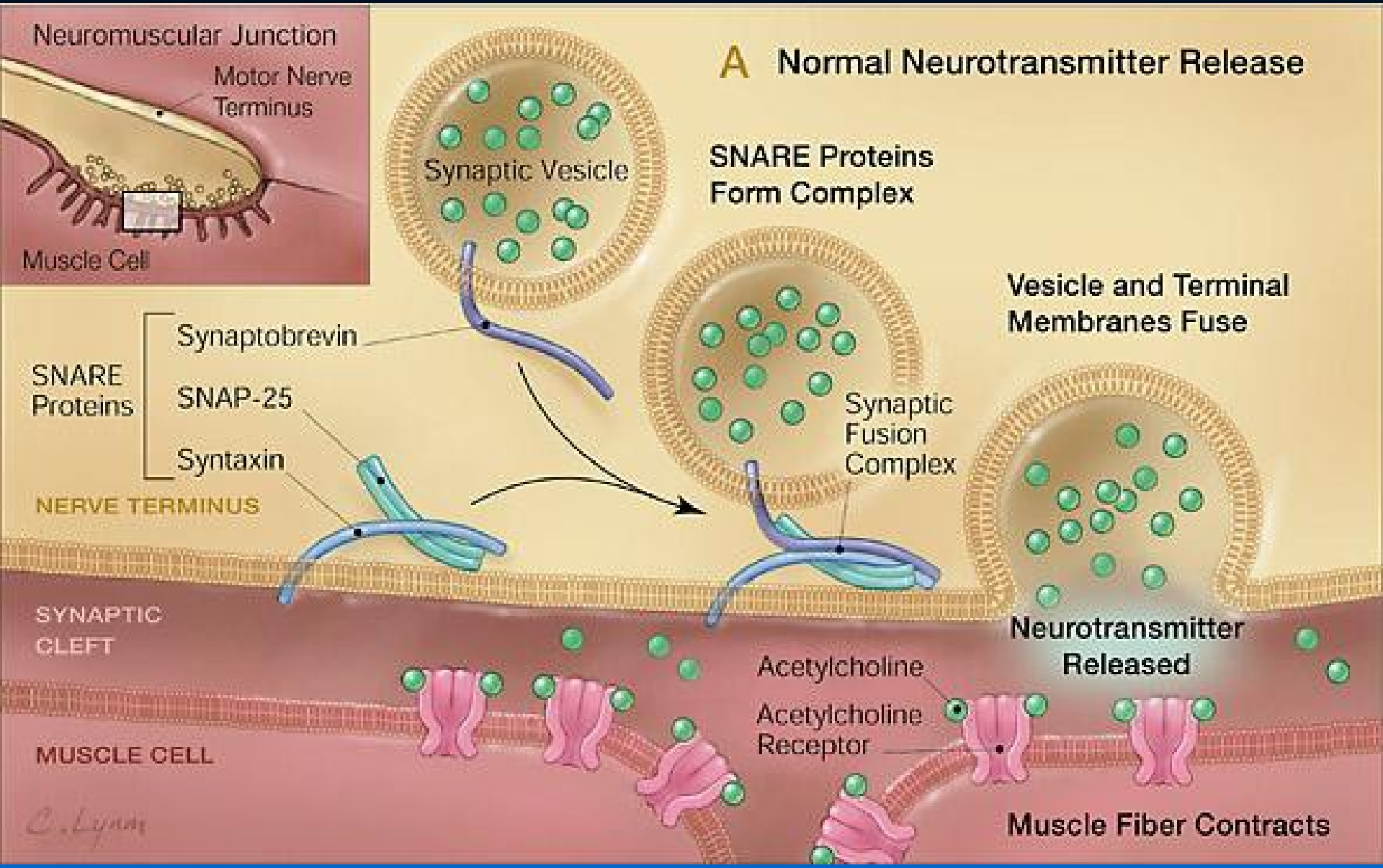

Botulinum neurotoxin

B Exposure to Botulinum Toxin

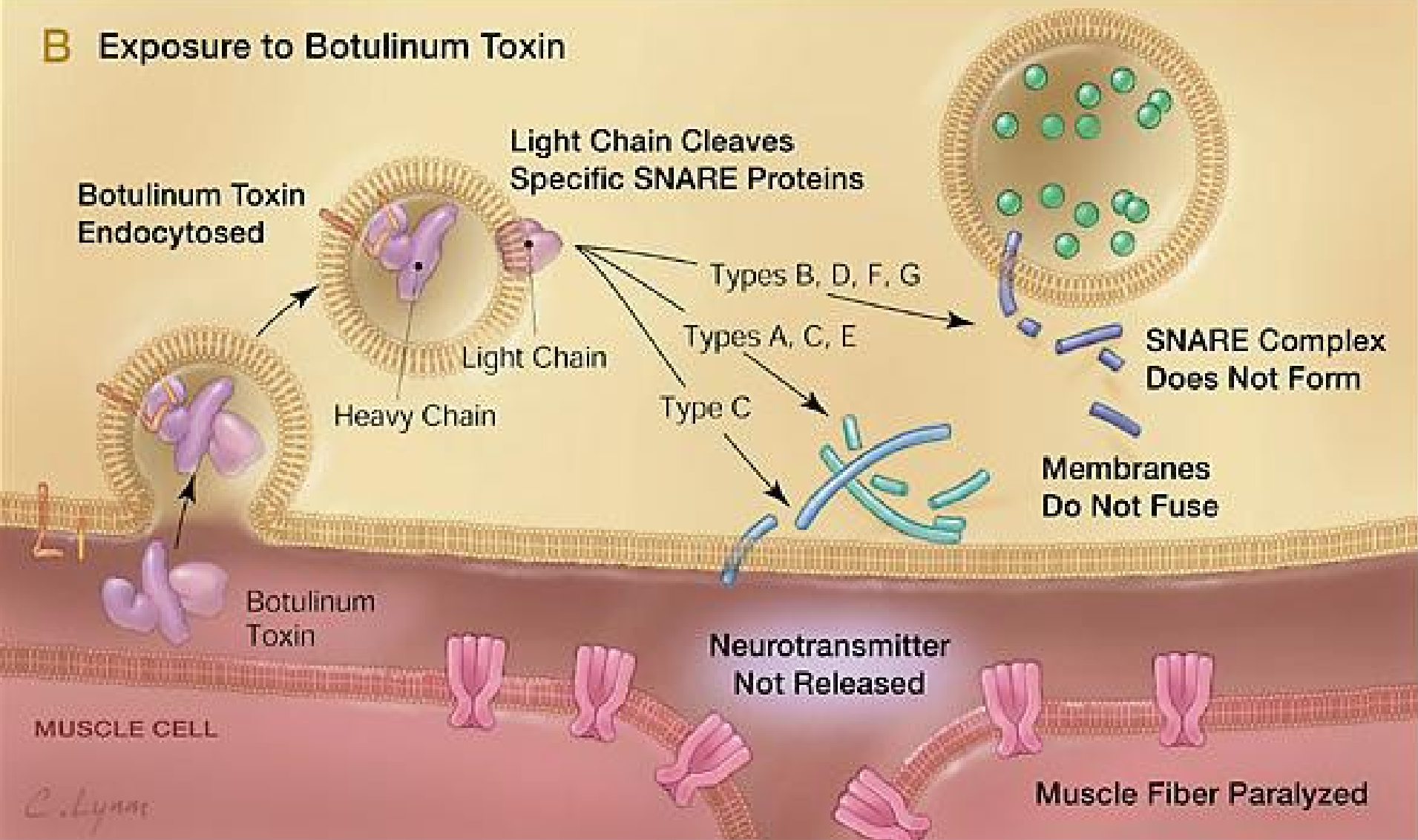

Botulism

Sentinel (Level A) laboratory procedures are designed to ensure proper:

- **Collection**
- **Packaging and shipping**
- **Safety**

Level A Biosafety Alert

- **Minute quantities acquired by ingestion, inhalation, or by absorption can cause profound intoxication and death**
- **Materials suspected of containing botulism toxin must be handled as BSL-2 specimen**

Laboratory-acquired cases:

There has been one report of 3 cases of laboratory-associated inhalation botulism

Holzer E. Botulism Caused by Inhalation. *Med. Klinik.* 1962;No. 41:1735-1740.

Safe Work Practices

- Minute quantities of toxin are hazardous
- Decontamination
 - 0.1N sodium hydroxide
 - Will inactivate toxin
 - 10% household bleach (prepared fresh daily)
 - Will inactivate toxin
 - Will kill vegetative cells and spores
 - Treat spills sequentially (15 to 20 minutes each) with sodium hydroxide, bleach solution, and finally isopropyl alcohol (to reduce caustic effects of decontamination procedure)

Safe Work Practices

- **Waste handling**

- All material with potential contact with botulinum toxin and/or *C. botulinum* must be autoclaved for 60 minutes, 121C, at 15 to 20 PSI
- Since small quantities of toxin may cause illness, all material in the laboratory should be considered contaminated

Safe Work Practices

- **Response to potential exposure**
 - **Workers should be made aware of early symptoms of botulism**
 - **Blurred or double vision**
 - **Dry mouth**
 - **Slurred speech**
 - **Peripheral muscle weakness**
 - **Self-monitor 2 to 4 days for symptoms**
 - **Report to emergency care facility if symptoms develop**
 - **Prophylactic antitoxin is not administered in the absence of symptoms**

Botulinum Toxoid Vaccine


- “Investigational New Drug”- for the past 30+ years
- Available for laboratory workers through your health clinic after enrollment with CDC Drug Services (404 639-3356)
- Initial series: 0, 2 weeks, 12 weeks, 1 year
- Booster provided every 2 years following proof of need (serum submission to CDC for residual antitoxin test prior to boost)
- May eliminate future treatment options of therapeutic toxin preparations.

Botulism

MDL Specimens

- 1. Pre-antitoxin serum - no anticoagulant (~15mL)**
- 2. Post-antitoxin serum – 6-12 h after antitoxin (~15mL)**
- 3. Other:**
 - 1. Stool-unpreserved (25g), Enema-sterile (25 mL)**
 - 2. Gastric aspirate or vomitus (25 mL)**
 - 3. Tissue or exudates (Port-A-Cul vials)**
 - 4. Food, Soil (50-100g), Water (\geq 100 mL)**

Laboratory Capacity for Botulinum Toxin Testing

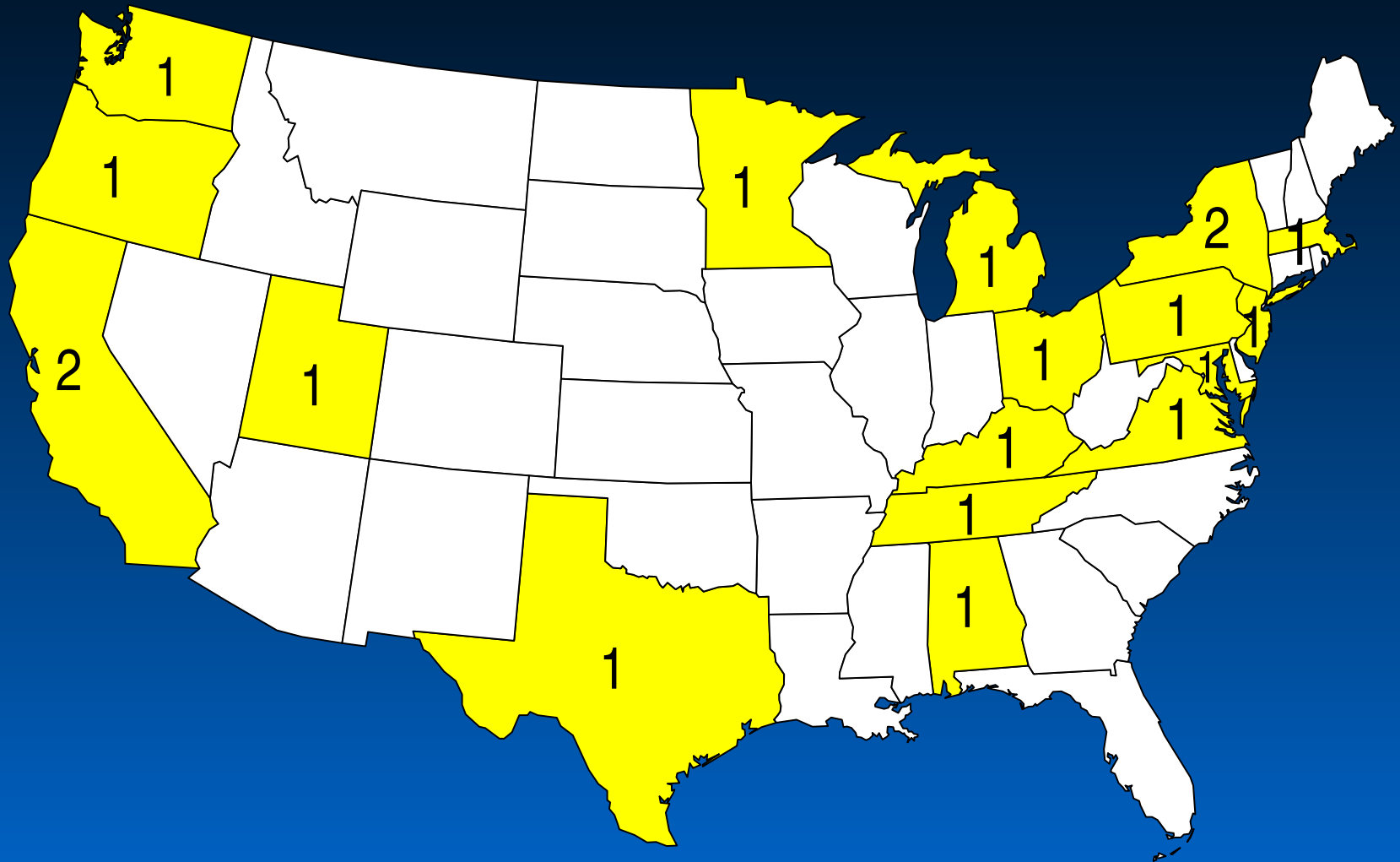

Current approved test

- **Mouse bioassay is the only currently approved test for the laboratory confirmation of botulism**
- **An ELISA (FDA/CDC) was recently validated for toxin detection in cultures**
 - **Currently under evaluation at CDC for utility in the clinical diagnostic laboratory**

Mouse Bioassay




Advantages

- Exquisitely sensitive (10 X 10^{-12} grams = 0.0000000000010 g)

Disadvantages

- Hazardous
- Extensive animal use
- Only 20 test sites around the country (17 state, 2 local, CDC)

Basis for Botulinum Toxin Detection

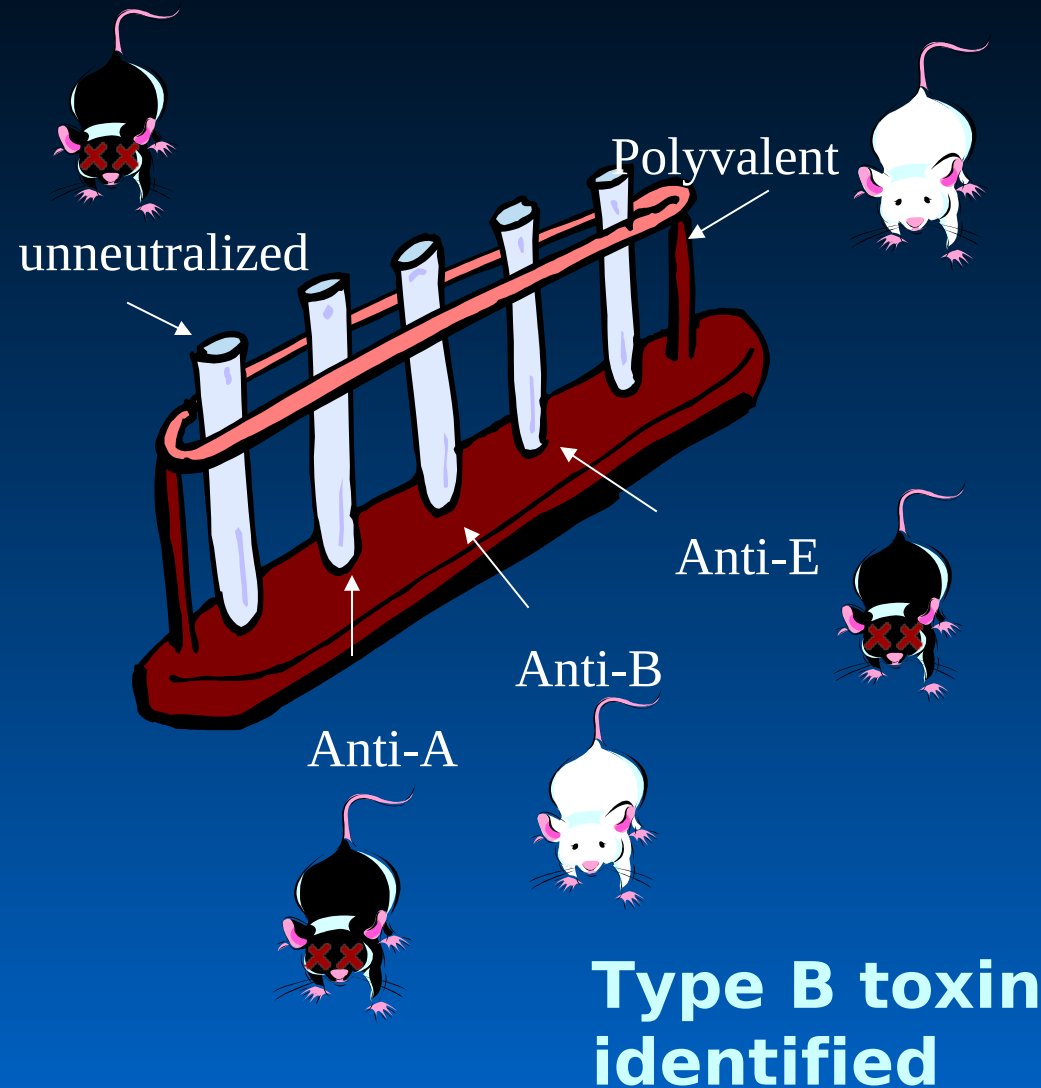

- Preparation of extract
- IP injection into 2 mice
 - Unneutralized
 - Neutralized with specific antitoxin
- 4 day monitor of mice for symptoms

Botulism

Packaging and Shipping

- **Consult your local public health lab for assistance**
- **Sterile leakproof container**
- **Insulated shipping containers with refrigerant (cold packs) 4-10°C**
- **Labeled “MEDICAL EMERGENCY, BIOLOGICAL HAZARD, REFRIGERATE ON ARRIVAL”**
- **Do not send USPS**

Botulism

Packaging and Shipping

- Ship as hazardous materials
- Do not freeze or expose to heat
- Notify the laboratory receiving the specimens prior to shipment
- Forward patient history and clinical symptom information with specimens
 - Notify the lab of any medication that the patient is taking which may interfere with testing

**REFER SPECIMENS TO THE
MICROBIAL DISEASES LABORATORY
(Richmond Campus)**

OR

**LOS ANGELES COUNTY PUBLIC HEALTH
LABORATORY
(if located in Los Angeles)**

Your Laboratory Response Plan

- **Develop and implement a bioterrorism response plan, test it, and update as needed**
- **Train your staff**
- **Know whom to call**
- **Know how and where to ship**
- **On advice of law enforcement —provide a chain of custody**

Role of the Sentinel (Level A) Laboratory

Rule out critical biological agents

OR

Refer to higher level laboratory



QUESTIONS?