

 <p><b>STOP Notification</b></p>	<h2 style="text-align: center;">Vortex Specimens Before Testing Aptima SARS-CoV-2 on the Hologic Panther</h2>							
<p><b>When:</b></p>	<p style="text-align: center;"><b>Wednesday, June 24, 2020</b></p>							
<p><b>What?</b></p>	<p><b>Vortex Aptima SARS-CoV-2 specimens 20-30 seconds prior to testing. Follow procedure for repeating Invalid results.</b></p>							
<p><b>Who is Affected:</b></p>	<p style="text-align: center;"><b>Affected Location(s)</b></p> <p>Alamance Cancer Center                  High Point Cancer Center                  Wesley Long Cancer Center                  Alamance Regional                  Annie Penn Hospital                  MedCenter @ High Point                  MedCenter @ Mebane                  X Moses Cone Hospital                  Wesley Long Hospital                  Women's Hospital</p>	<p style="text-align: center;"><b>Department(s)</b></p> <p>Blood Bank                  X Cytology                  Flow Cytometry                  Histology                  X Microbiology                  Phlebotomy                  Point Of Care                  X Rapid Response Lab                  Respiratory Therapy                  X Specimen Processing</p>						
<p><b>Why?</b></p>	<p><b>To reduce the number of initial invalid SARS-CoV-2 test results on the Hologic Panther</b></p>							
<p><b>What you will need to do to prepare:</b></p>	<p><b>Vortex specimens 20-30 seconds before running and follow instructions in procedure on repeating samples with Invalid results.</b></p>							
<p><b>Manager / Supervisor Responsibility:</b></p>	<p>Send Post Live Documentation to IT Manager within 5 days of the effective date for test systems marked with an "X"</p> <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="width: 50%; text-align: center;">Sunquest</td> <td style="width: 50%; text-align: center;">WindowPath</td> </tr> <tr> <td style="text-align: center;">CHL</td> <td style="text-align: center;">Instrument / Manual Test</td> </tr> <tr> <td style="text-align: center;">PowerPath</td> <td style="background-color: #cccccc;"></td> </tr> </table>		Sunquest	WindowPath	CHL	Instrument / Manual Test	PowerPath	
Sunquest	WindowPath							
CHL	Instrument / Manual Test							
PowerPath								
<p><b>Need Help?</b></p>	<p style="text-align: center;">Contact your Manager / Supervisor</p>							