

Procedure Change/Revision (IMM 36-A)

Procedure: IgA on the BNII Nephelometer (Doc# IMM 184)

Change/Revision:

Section I. Interpretation of Results add the following:

4. Monitor the pending report for test CECHK

CECHK test added to all CESER (Celiac Serology Panel) or CERFX (Celiac Reflexive Panel) orders:

CESER: Is ENDOA is collected and received?

YES

1. No further action is needed.
2. Dot CECHK and verify.

NO

1. Collect and Receive ENDOA.
2. Print MAYOI Aliquot label.
3. Pour aliquot and place in MAYO refrig rack.
4. Dot CECHK and verify.

CERFX: Is IgA level ≥ 7 mg/dL

Yes

1. Print DSX label
2. Place is DSX rack for TTGA and GLIA
3. Dot CECHK and verify.

NO

1. Collect and receive TTGG.
2. Print TTGG label and DSX label.
3. Use CECHK tube for TTTG and IGA tube for DSX.
4. Place TTTG tube in ARUP freezer rack. Place DSX tube in DSX rack
5. Dot CECHK and verify.

Date of Revision: 1/30/13

Director Signature/Date:

B. H. M. 1/31/2013

Technologist Signature/Date: SEE MTS