

Procedure Change/Revision (IMM 36-A)

Procedure: Specimen Processing Procedure (Doc# IMM 145)

Change/Revision:

Section IX SPECIAL HANDLING PROTOCOLS, add the following.

K. Celiac Panels

1. Celiac Serology Panel: Always Includes the following:

IgA (IGA), TTG IgA (TTGA), Gliadin IgA (GLIA), Gliadin IgG (GLIG) and Endomysial Ab (ENDOA)

2. Celiac Disease Reflexive Panel: IgA is performed

If IgA \geq 7 mg/dL:

TTG IgA and Gliadin IgA (both performed on the DSX)

If IgA < 7 mg/dL:

TTG IgG (ARUP) and Gliadin IgG (DSX)

CECHK test added to all CESER or CERFX orders.

Label prints when Immunology label is tracked to the Aliquot Bench.

For CESER (Celiac Serology Panel)

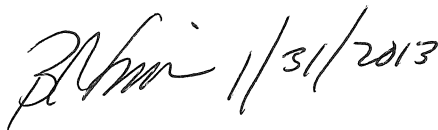
1. Collect and Receive ENDOA
2. Print MAYOI Aliquot label
3. Pour aliquot

For CERFX (Celiac Reflexive Panel)

1. Use the CECHK label to make an aliquot tube
2. Give aliquot tube to the BNII tech.

Date of Revision: 1/30/13

Director Signature/Date:

 1/31/2013

Technologist Signature/Date: SEE MTS