

Procedure Change/Revision

(IMM 163, IMM166, IMM167, and IMM165)

Procedure: Quantalyzer Daily Maintenance

Change/Revision: Perform Washer Dispense and Washer Aspirate Verification during daily maintenance. Follow prompts on Quantalyzer Maintenance screen – load an ELISA plate on plate 1 area.

- 1. Do dispense verification first-after the dispense verification is done, remove the plate and examine for even dispensing of CLRW in all wells.**
- 2. Next place the plate back into plate 1 area. Perform the Aspirate verification test. After the aspirate verification test is complete, remove the plate and examine that all wells have been completely aspirated.**
- 3. If any of the above conditions fail, take a metal stylus from DSX equipment and clean the aspirate/dispense ports on the Quantalyzer wash head.**
- 4. Then repeat Steps 1 and 2.**

Date of Revision: 2/15/2013

Director Signature/Date:

Blum
2/20/2013

Technologist Signature/Date: