

Procedure Change/Revision (IMM 36-C)

Procedure: Tosoh A1A 900 (Doc# IMM 200)

Change/Revision: Quarterly Maintenance-Decontamination of Wash and Diluent lines.

Part B. Add the following additional steps:

3. After bleach step #2 has been completed, place a large beaker of CLRW and rinse the wash and diluent lines thoroughly.

- a. Place new tank filters onto the end of the lines and replace both Wash and Diluent lines into their respective bottles.**
- b. Next under the Maintenance program, repeat steps 5 and 6 each five times. This will replace all the lines with diluent and wash and removes all the bleach solution. Empty the liquid waste container and rinse thoroughly until all bleach odor has disappeared.**

Date of Revision: 3/1/13

Director Signature/Date:

(Signature) 3/1/13

Technologist Signature/Date: