

Procedure Change/Revision (IMM 36-A)

Procedure: Kappa and Lambda Free Light Chains (Binding Site) on the Siemens BNII Nephelometer (Doc# IMM 199)

Change/Revision: In the Interpretation of Results Section, Part A Reporting Results

- 1. The following change was made regarding retesting of New patients with abnormal results:**

If the patient does not have any results in Test History, **and the Kappa, Lambda or Kappa/Lambda Ratio is out of the normal range**, repeat the testing at a 1:400 dilution.

- Go to Lab Journal
- Open the sample by double clicking on the Order#
- Click Dilution
- Click on 1:400, then OK
- Reload the sample on the BNII

If the 1:400 result is less than **5 times** 1:100 result, report the 1:100

If the 1:400 result is greater than 5 times 1:100 result, report the 1:400.

- 2. K/L Ratios less than 0.01 will be reported as <0.01 by SOFT. When the Kappa is less than the AMR and the ratio is <0.01, manually enter the ratio as <0.01.**

Date of Revision: 3/8/13

Director Signature/Date:

Brian R. Smith MS 3/8/13

Technologist Signature/Date: SEE MTS