


Procedure Change/Revision (IMM 36-A)

Procedure: Anti Double Stranded DNA Antibody Detection by IFA Using Bio-Rad Kallestad Crithidia Licilae (Doc# IMM 148)

Change/Revision: Reading: using Leica DM 4000 B Microscope (SN# 0620172412)

1. Turn the shutter power ON
2. Adjust the shutter intensity to four (second to the last)
3. Turn the Leica microscope power ON
4. Turn the microscope's FL shutter ON
5. Looking at the display, make sure that you are using 40x objective and the FIM (fluorescence) is at 10%

Date of Revision: March 7, 2013

Director Signature/Date: 

Technologist Signature/Date: