

# Procedure Change/Revision (IMM 36-C)

**Procedure:** Specimen Processing Procedure (Doc# IMM 145)

**Change/Revision:** Processing of *Aspergillus Galactomannan* will be performed by Immunology as follows:

1. Red top tube received with Micro label
2. Spin at 3000 rpm for 15 min.
3. Label 15ml conical tube with Micro label.  
(place label vertically on tube)
4. Aliquot serum immediately after uncapping red top tube and cap aliquot tube.
5. Place aliquot in *Aspergillus* rack in freezer.

**DO NOT PLACE THESE SAMPLES ON THE TECAN**

**Date of Revision:** 3/26/12

**Director Signature/Date:**

*PO 3/26/12*

**Technologist Signature/Date:**