

# Procedure Change/Revision (IMM 36-C)

**Procedure:**

C2729

**Change/Revision:**

- 1.) For serial dilutions, program sample using nine digit order number, followed by a hyphen, then dilution factor. For example if order number is 272202646 and dilution factor is 84, enter as "272202646-84"
- 2.) For serial dilutions, retrieve value from printout and enter in 2729I field on Results tab in Instrument Menu on the original sequence. Correct the dilution factor and Verify All.
- 3.) If a sample barcode fails to read, indicated as "SAMPLE1" in Instrument Menu, retrieve value from printout and calculate manually in Result Entry.

**Date of Revision:**

3/23/2012

**Director Signature/Date:**

  
3/26/2012

**Technologist Signature/Date:**



Results (40/40)

TAT Status	Order	Flags	Seq #	Time	T #	Tests list
[icon]	B1	VQ	0001	12:57	1	VB2MI
[icon]	B2	VQ	0002	12:59	1	VB2MI
[icon]	B3	VQ	0003	13:00	1	VB2MI
[icon]	272203753-OM	V	0004	13:02	1	VB2MI
[icon]	272203769-OM	V	0005	13:04	1	VB2MI
[icon]	272201896-OM	V	0006	13:05	1	VB2MI
[icon]	272201493-OM	V	0007	13:07	1	VB2MI
[icon]	C1	VQ	0008	13:09	1	V2729I
[icon]	C2	VQ	0009	13:10	1	V2729I
[icon]	C3	VQ	0010	13:12	1	V2729I
[icon]	272300551-OM	V	0011	13:14	1	V2729I
[icon]	272203759-OM	V	0012	13:15	1	V2729I
[icon]	272203151-OM	V	0013	13:17	1	V2729I
[icon]	272203812-OM	V	0014	13:19	1	V2729I
[icon]	272202641-OM	V	0015	13:20	1	V2729I
[icon]	272203165-OM	V	0016	13:22	1	V2729I
[icon]	272202646-OM	V	0017	13:24	1	2729I
[icon]	272202846-OM	V	0018	13:25	1	V2729I
[icon]	272202990-OM	V	0019	13:27	1	V2729I
[icon]	272202717-OM	V	0020	13:29	1	V2729I
[icon]	272202719-OM	V	0021	13:30	1	V2729I
[icon]	272202843-OM	V	0022	13:32	1	V2729I
[icon]	272202859-OM	V	0023	13:34	1	V2729I
[icon]	272203095-OM	V	0024	13:35	1	V2729I
[icon]	272203221-OM	V	0025	13:37	1	V2729I
[icon]	272203815-OM	V	0026	14:25	1	V2729I
[icon]	272203787-OM	V	0027	14:26	1	V2729I
[icon]	272203792-OM	V	0028	14:28	1	V2729I
[icon]	272203706-OM	V	0029	14:30	1	V2729I
[icon]	272203794-OM	V	0030	14:31	1	V2729I
[icon]	272203806-OM	V	0031	14:33	1	V2729I
[icon]	272203811-OM	V	0032	14:35	1	V2729I
[icon]	272203789-OM	V	0033	14:36	1	V2729I
[icon]	272203160-OM	V	0034	14:38	1	V2729I
[icon]	272203704-OM	V	0035	14:40	1	V2729I
[icon]	272203795-OM	V	0036	14:41	1	V2729I
[icon]	SAMPLE1	Q	0037	14:43	1	2729I
[icon]	272202646-42	Q	0038	14:45	1	2729I
[icon]	272202646-84	Q	0039	14:46	1	2729I
[icon]	272202646-168	Q	0040	14:48	1	2729I

Filter List Ord Seq All Stat

Patient Info

Name (last, first, middle):

MRN: Sex: Age: Ward: Room: MOB:

Order Info

Tray Tube: SAMPLE1 Prior: Seq: 0037 Spec: IR8 Collected: 03/22/2012

Tests: Call Comment(s) Sp

QC Post All Tests QC Post Selected Test Toggle Cancel Status Bridge to QC Label

TAT Status	Stst	Result	FI	Flags	D	Error	Status	Tech	M	Wkst	Prev Res
1	[icon]	2729I	16.60				O Q				

Cancer Antigen 27.29 Value

Instrument Lab Results