

Procedure Change/Revision (IMM 36-C)

Procedure:

B2M

Change/Revision:

- 1.) For serial dilutions, program sample using nine digit order number, followed by a hyphen, then dilution factor. For example if order number is 272202646 and dilution factor is 102, enter as "272202646-102"
- 2.) For serial dilutions, retrieve value from printout and enter in B2MI field on Results tab in Instrument Menu on the original sequence. Correct the dilution factor and Verify All.
- 3.) If a sample barcode fails to read, indicated as "SAMPLE1" in Instrument Menu, retrieve value from printout and calculate manually in Result Entry.

Date of Revision:

3/23/2012

Director Signature/Date:

[Handwritten Signature]
3/26/2012

Technologist Signature/Date: