

Procedure Change/Revision (IMM 36-A)

Procedure: Lyme Western Blot IgG and IgM –MarDx Trinity Biotech, DOCUMENT # Imm 168

Change/Revision:

Section VII,

- 1, Revision: Request a worklist by Test: LYWB and LYME; record Lyme ELISA result for each patient; investigate / address any ordering issues based on the result (Western Blot on a LYME negative specimen or absence of ELISA result.
- 1, Addition: In Specimen Receiving, print 2 labels for each patient, One each for IgG & IgM and affix to respective worksheets
- 2, Addition: Strips should be used in numerical order (sequentially). When a new vial of strips is opened, used extra tubes and separate strips in sets of ten, again in sequential order.

Section VII, Addition:

6. Record results of Negative and Serum Band Locator on QC chart on L Drive

Section IX: eliminate instructions on interpretation of assay as this will now be performed by Resident/Attending. Technologist will simply record presence of bands on worksheets.

Section X:

Revise to eliminate the entry of Test Interpretation.
Insert instructions to perform Test Review Entry

Date of Revision: 6/6/12

Director Signature/Date:



Technologist Signature/Date:
