



**LABORATORY ENVIRONMENT  
SUPPORT SYSTEMS, INC.**

1 (800) 621-6404

## ACL TOP® CUVETTE STRIPS

### RINSING, PACKING AND SHIPPING PROCEDURES For Our Standard Volume Users

#### Items Supplied

- 1 – Shipping Container with Lid & Drain Lid
- 2 – Bottles of Decontamination Solution
  - 1 – Ziplock Liner Bag
  - 1 – Plastic Cover Tray
- 1 – FedEx Return Record

#### 1. PREPARE

Fill the empty container with approximately 4 liters of DI water or tap water and add 1/2 bottle of the decontamination solution provided. Additional decontamination solution provided if higher concentration desired.

#### 2. DECONTAMINATE


Unscrew the lid on the shipping container. Add the cuvettes to the decontamination solution, replace the lid and tighten snugly. After adding cuvettes, replace the solid lid firmly in place and rotate the container to allow the decontamination solution to come into contact with the new addition of cuvettes. Repeat this procedure until the container is full. Once the container is full, the drain lid may be exchanged for the solid lid and the container placed on its side adjacent to the sink to allow the rinsing solution to drain into the sink. Retain drain lid for future use.

#### 3. BOX AND SHIP

When the container is full (it will hold approximately 750 cuvette strips), place the container in the ziplock bag provided and place in the shipping box on its side. Place up to 12 empty cuvette boxes in the adjacent compartment and fit the cover tray over these items. Enclose a completed FedEx Ground Pickup Record and empty solution bottle. Seal box with preprinted return address on outside flap and call FedEx Ground at 1-800-463-3339 for pick up.

*ACL TOP® is a registered trademark of Instrumentation Laboratory.*

REV-8-2013

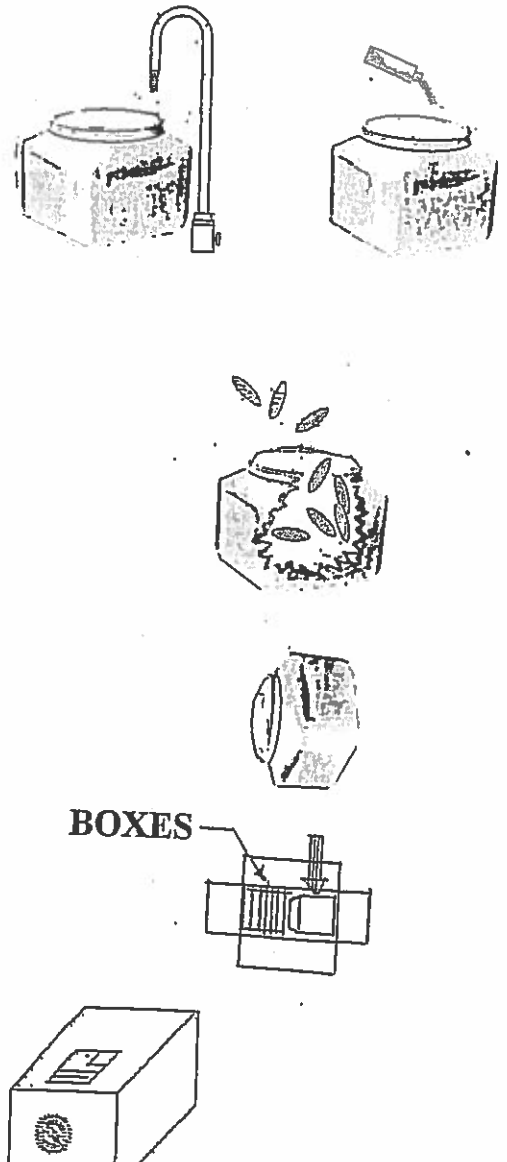

BOXES

