



Cobas e411 Maintenance Log

**CHEM.COBAS.2.2 Cobas e411 Maintenance Log** Site: \_\_\_\_\_ Month/Year \_\_\_\_\_

<b>Daily</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
Clean S/R Probe																																
Check condensation inside reagent rotor and system reagent compartments																																
Turn operator switch on																																
Empty liquid waste																																
Empty solid waste																																
Check water (fill as needed)																																
Check tips and cups																																
Open ProCell and CleanCell																																
Check reagent/cal status																																
Run QC																																
Clear sample data (end of day)																																
Perform finalization (end of day)																																
Put in "sleep" mode (end of day)																																
<b>Tech</b>																																

<b>Weekly</b>	<b>Date/Tech</b>						
Clean incubator and aspiration station							
Clean sipper probe							
Clean flow path for the direct drain							

Supervisor review: \_\_\_\_\_



Cobas e411 Maintenance Log

Every 2 Weeks	Date/Tech
Clean rinse station	

Monthly	Date/Tech
Perform liquid flow cleaning	

As Needed	Date/Tech

Supervisor review: \_\_\_\_\_