

**Document Title: TEG Graphical Depictions  
SH.CP.AU.jad.0158.0001**

<b>Author</b>	<b>Effective Date:</b>	<b>Supersedes Procedure #</b>
Zachary Boldt		new

<b>Revised by:</b>	<b>Date Revised:</b>	<b>Effective Date:</b>

<b>Approval Signature</b>	<b>Approval Date</b>
Majed Refaai, M.D., Lab Director	

<b>Distributed to</b>	<b># of Copies</b>	<b>Distributed to</b>	<b># of Copies</b>
Laboratory Room G1564	1	QC/QA	1
Sharepoint	Electronic		

**REVISION HISTORY**



<b>Procedure #</b>	<b>Revision Date</b>	<b>Reason for Revision</b>

QC

L1 – Level I control

Reconstitute



1 ml dH<sub>2</sub>O

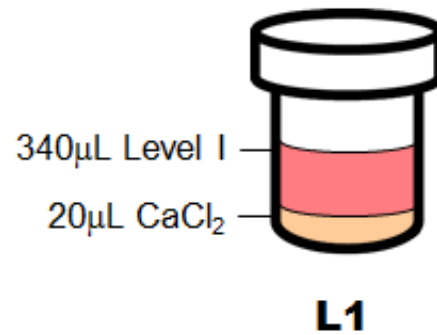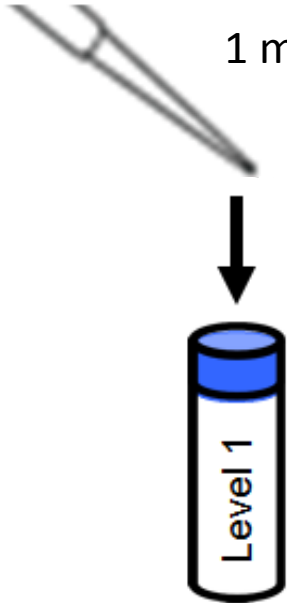

L2 – Level II control


Reconstitute

1 ml dH<sub>2</sub>O

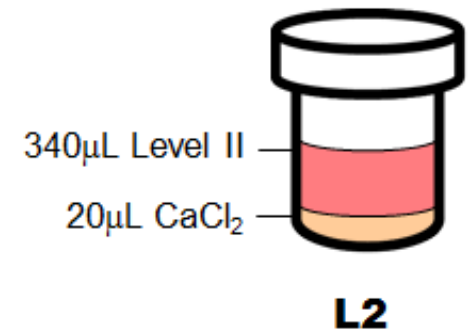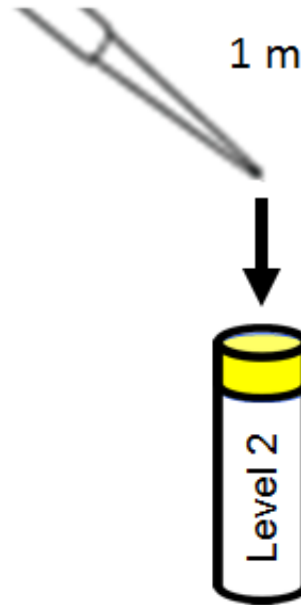


# TEG


## CK – Citrated Kaolin

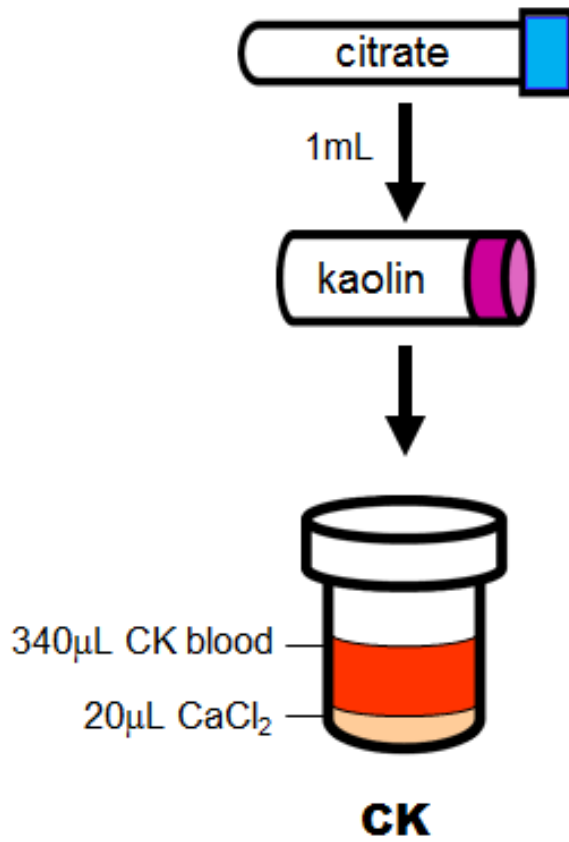


## CKH – Citrated Kaolin Heparinase

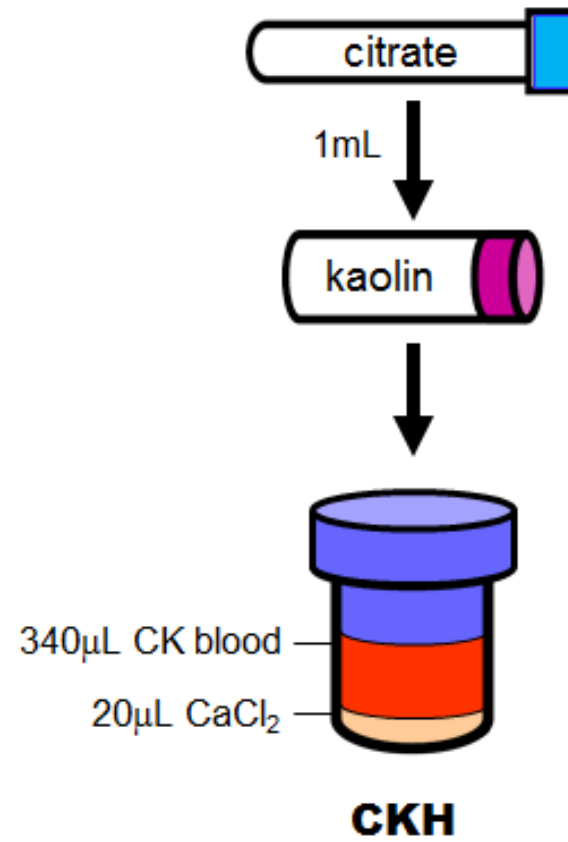

# Platelet Mapping

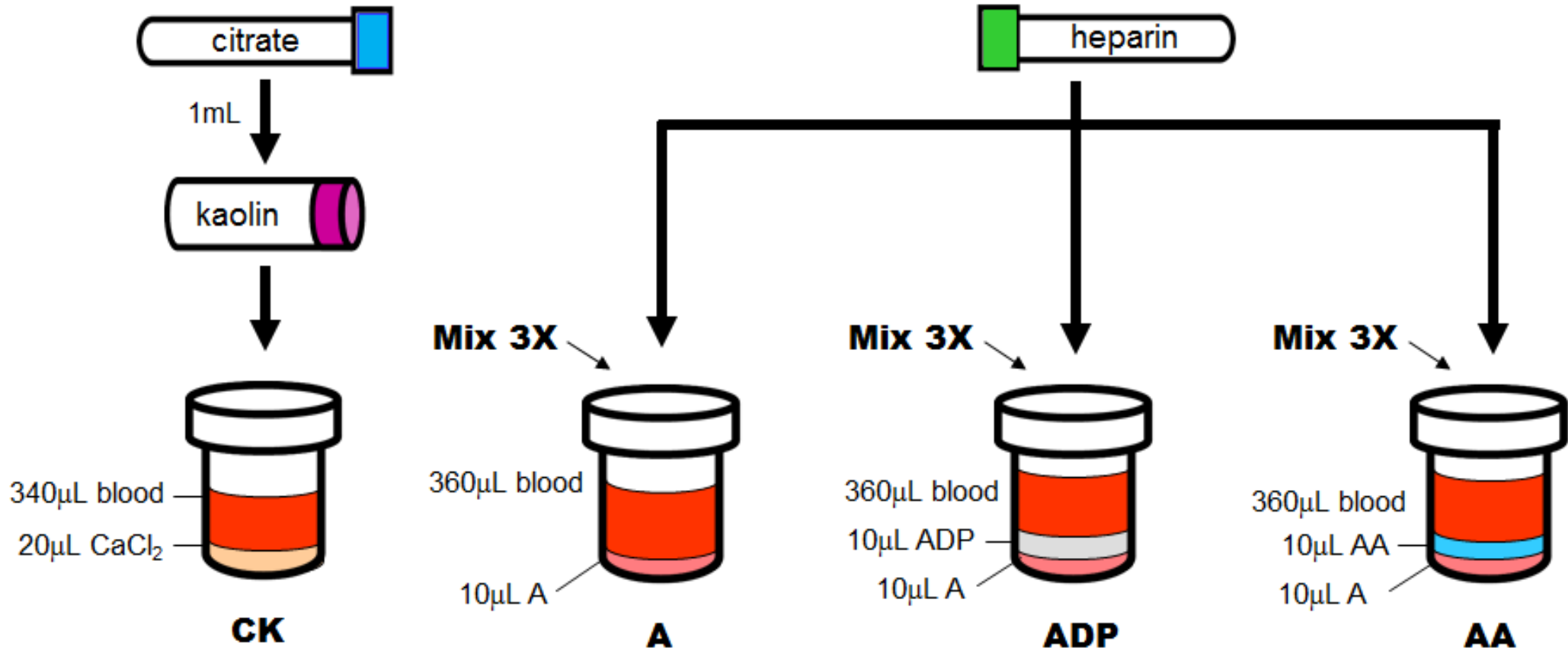

# Heparinase Platelet Mapping

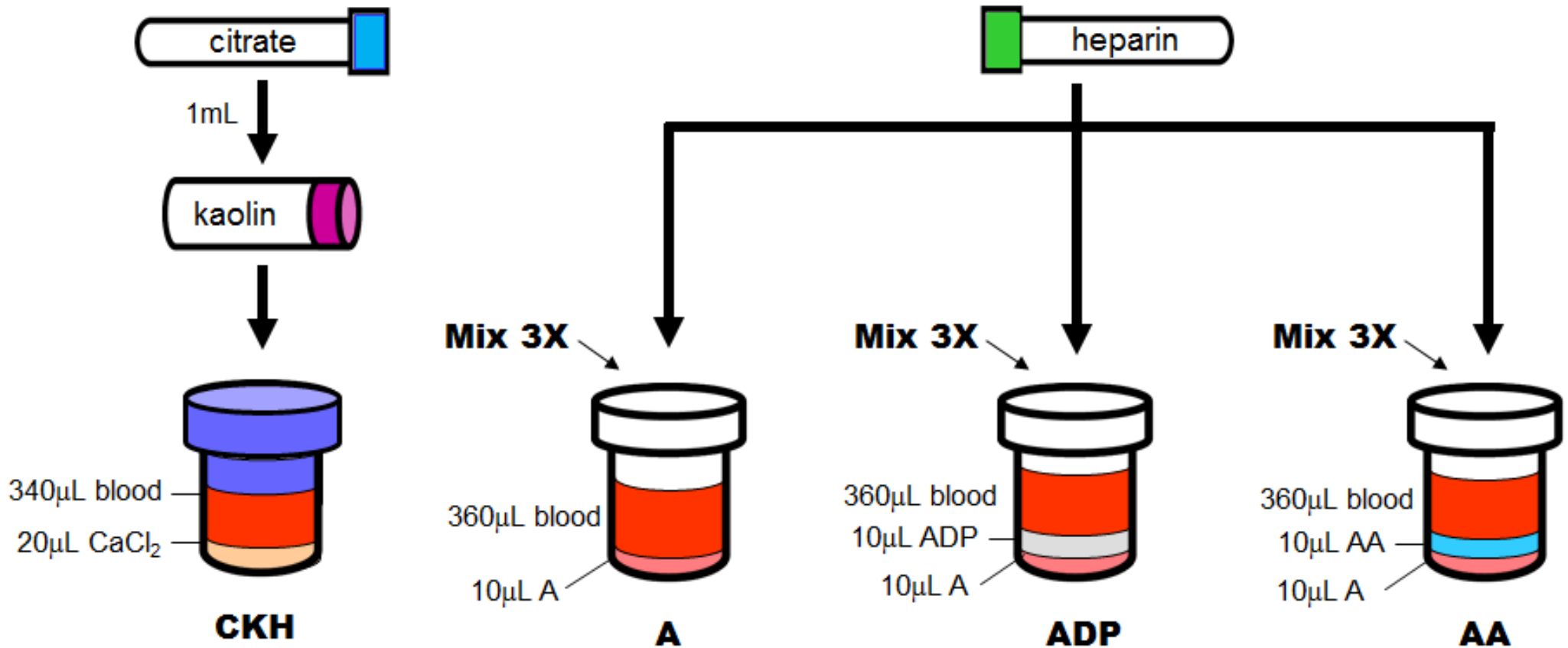

# Functional Fibrinogen

## Watch for Growing MA:

Once you have started all channels, **DO NOT DISCARD Activator**. Monitor the Activator (A-P1) channel. It should resemble Figure 2 (top and bottom lines are parallel).

If the tracing looks like Figure 1:

**MA<sub>A</sub> is greater than 25mm AND the TMA is greater than 15 minutes**

Run the Functional Fibrinogen

Occasionally you will get a tracing that looks like figure 1 but, the MA finalizes before platelet activation (secondary clot growth) begins. In this case it is not necessary to use FF as it will not change the result.




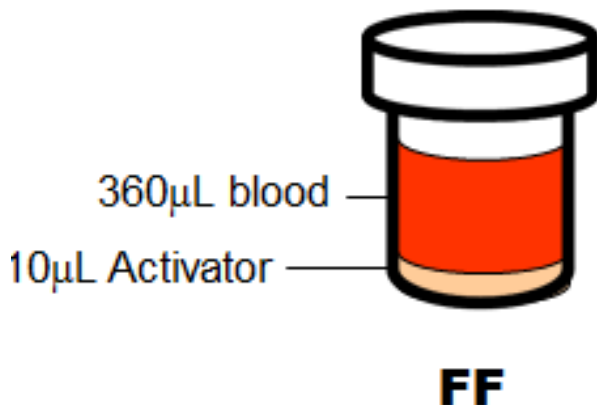


Figure 1

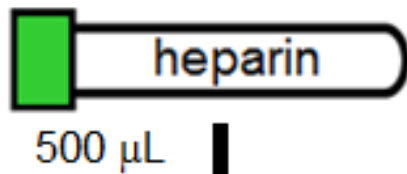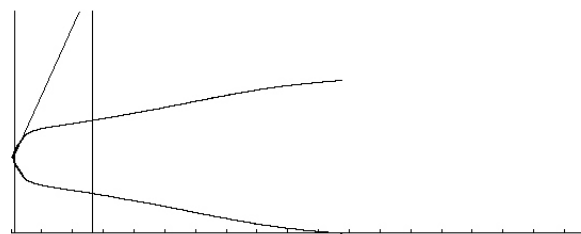

R	K	Angle	MA	LY30	G
min	min	deg	mm	%	d/sc
0.5	12.7	62.0	41.5	*0*	3.6K

Figure 2

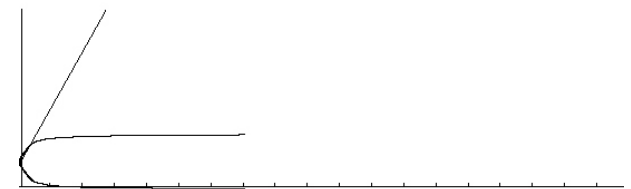

R	K	Angle	MA	LY30	G
min	min	deg	mm	%	d/sc
0.5	N/A	56.2	12.8	0.0	0.7K