

F.Y.I.

**Attention
MRI & CT Staff**

Point of Care STATSensor Testing Workflow *(Effective: Immediately)*

As you are aware, CT & MRI now requires operators to scan a patient's barcode armband when they perform a STATSensor Creatinine. Scanning the correct barcode is important.

1. We now require that the users **SCAN ONE OF THE THREE SMALLER 2D BARCODES** (see below) and not the larger 2D barcode.
2. If the STATSensor pulls up the patient name proceed with testing. If no name appears ensure that correct HAR was used to generate the barcode armband. If not correct, request a new armband.
3. If the correct barcode armband is used, and the patient name does not appear on the screen, select "Override" and manually result test in EPIC.



Please contact your site Point of Care Coordinator with any questions

MD 05/12/17