

Name: Iron

Type: Special
Modified: 2003/12/22

Origin: Factory Released
Author: DakoCytomation Cambridge

Description: Stains Hemosiderin

Step	Dispense Liquid	Volume (mL)	Time (secs)	Heater (C)	Mixes	Aspirate	Waste
001	None	-	0	Off	0	On	Water Soluble Waste
002	Potassium Ferrocyanide - Iron	1.00	0	Off	0	Off	
003	Hydrochloric Acid - Iron	1.00	260	37	4	Off	
W1	Wash Solution	3.00				On	Combined Dye Waste
W2	Wash Solution	2.00				On	Combined Dye Waste
W3	Wash Solution	3.00				On	Combined Dye Waste
WF	Wash Solution	3.00				On	Combined Dye Waste
004	Nuclear Fast Red - Iron	1.00	280	37	1	Off	
W1	Wash Solution	3.00				On	Combined Dye Waste
W2	Wash Solution	2.00				On	Combined Dye Waste
W3	Wash Solution	3.00				On	Combined Dye Waste
W4	Wash Solution	2.00				On	Combined Dye Waste
W5	Wash Solution	3.00				On	Combined Dye Waste
WF	Wash Solution	3.00				On	Combined Dye Waste