

Name: Warthin-Starry (W-S)

Type: Special

Modified: 2004/03/03

Origin: Factory Released

Author: DakoCytomation Cambridge

Description: Stains Helicobacter pylori & Spirochetes

Step	Dispense Liquid	Volume (mL)	Time (secs)	Heater (C)	Mixes	Aspirate	Waste
001	None	-	0	Off	0	On	Water Soluble Waste
002	Pretreatment - W-S	1.00	110	60	2	Off	
003	Silver Nitrate - W-S	1.00	200	60	2	Off	
004	Hydroquinone - W-S	1.00	105	60	4	Off	
W1	Wash Solution	3.00				On	Trace Metal Waste
W2	Wash Solution	2.00				On	Trace Metal Waste
W3	Wash Solution	2.00				On	Trace Metal Waste
W4	Wash Solution	2.00				On	Trace Metal Waste
W5	Wash Solution	2.00				On	Trace Metal Waste
WF	Wash Solution	3.00				On	Trace Metal Waste