

7% BSA

ORTHO™ 7% BSA

REF

6844285
6986722

Reagents

ORTHO™ 7% BSA is an aqueous solution of bovine serum, inorganic salts, and preservatives.

Reactive Ingredients

None

Other Ingredients

7% bovine serum albumin, inorganic salts and preservatives

Warnings and Precautions

WARNING: While these products are bovine in origin, they should be handled using the same precautions as with any other blood or blood-derived product.

WARNING: The packaging (vial stopper) of this product contains dry natural rubber (latex), which may cause allergic reactions in some individuals.

Storage

Storage and Stability

Reagent	Storage Condition	Stability
Unopened	Frozen ≤ -18 °C	Until expiration date
	Refrigerated 2–8 °C	Until expiration date
Opened	Refrigerated 2–8 °C	≤ 28 days if tightly stoppered

Caution: Do not use visibly damaged product or product with incompletely sealed packaging. Product should appear clear, slightly yellow to yellowish-brown liquid, free of particulates.















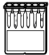


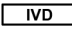






Materials Provided

12 vials (5 mL each) of ORTHO™ 7% BSA (Product Code 6844285)

15 vials (12 mL each) of ORTHO™ 7% BSA (Product Code 6986722)

Glossary of Symbols

The following symbols may have been used in the labeling of this product.

	Do Not Reuse		Contains Sufficient for "n" Tests		Fragile, Handle with Care.
	Use by or Expiration Date (Year-Month-Day)		<i>In vitro</i> Diagnostic Medical Device		Keep Dry
	Batch Code or Lot Number		Upper Limit of Temperature		This end up
	Serial Number		Lower Limit of Temperature		Do Not Use if Damaged
	Catalog Number or Product Code		Temperature Limitation		Cassettes/Cards
	Caution		Consult instructions for use		Concentration
	Date of Manufacture		Biological Risks		Der Grüne Punkt (the Green Dot). Manufacturer follows certain packaging material waste disposal management regulations.
	Manufacturer		Health Hazards		
	Authorized Representative in the European Community				

Revision History

Date of Revision	Version	Description of Technical Changes*
2015-08-31	2.0	<ul style="list-style-type: none"> Materials Provided: added new product code (6986722) Glossary of Symbols: added Globally Harmonized Symbol to comply with the Classification, Labelling and Packaging (CLP) Regulations
2014-07-10	1.0	Initial version.

* The change bars indicate the position of a technical amendment to the text with respect to the previous version of the document.



Ortho-Clinical Diagnostics, Inc.
100 Indigo Creek Drive
Rochester, NY 14626

Ortho Clinical Diagnostics

© Ortho-Clinical Diagnostics, Inc., 2014-2015.