

Standard Operating Procedure

SUBJECT: Temperature Indicator strips		
ORIGINATION DATE: 3/11/26	PREPARED BY:	APPROVED BY (If Applicable):
LAST REVIEWED: 3/2/2026	Lab	Venita Minnicozzi

PROCEDURE:

When to use:

1. Place on any product that is dispensed from lab.
2. DO NOT place indicators on general inventory.

How To use:

1. Remove Timestrip Blood temp 10 indicator from box.
2. Squeeze Red button till yellow box starts to turn green
3. Place on bottom portion of unit



How to Read:

1. Yellow: portion indicates temperature strip is inactive
2. Green: indicates indicator strip is active
3. White: Indicated Blood product is at an acceptable temperature
4. Partial Blue: Refrigerate product as soon as possible
5. Full Blue: Product is unable to be returned to inventory. Product must be brought back to the lab before being destroyed.

