


ABID – Weakly Reactive Antibody Screen (?/1+ ABSC)

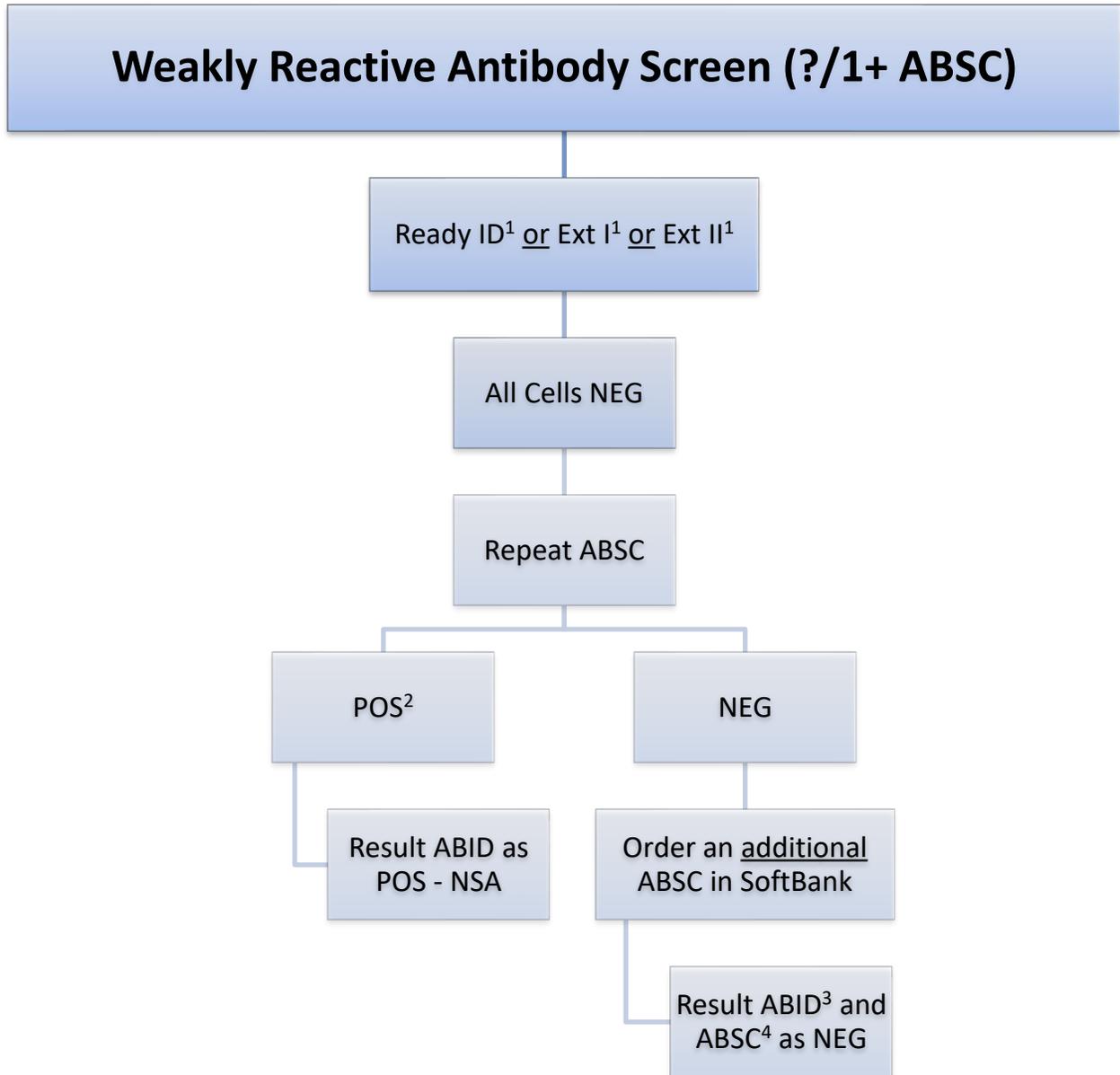

¹ Run appropriate panel; avoid running unnecessary panel(s). i.e. Run Ready ID for most patient; Run Ext II for Rh Neg patient if reaction(s) is on cell I or II.

² All clinically significant antibody were ruled out.

³ All ABID must be reviewed and resulted by a second tech.

⁴ By entering the NEG result on the additional ABSC, the status of the initial screen will change to “to be repeated”. A full panel must be ran on all positive initial ABSC.