



Binax NOW Legionella Urinary Antigen

Prepared By: Jolene Fair
Document Control Number: MB.40.010.00

Date: 05/10/2019

Approved By: *Scott Arnold*
Scott Arnold, MD

Date: 5/21/19

Daniel Buxton
Daniel Buxton, MD

Date: 5/21/19

Geoffrey Hahn
Geoffrey Hahn, MD

Date: 5/24/19

Ersie Pouagare
Ersie Pouagare, MD

Date: 5/28/19

Date: _____

Adopted:

Date: June 1, 2019

Reviewed:		Date:	
Reviewed:		Date:	
Reviewed:		Date:	
Reviewed:		Date:	
Reviewed:		Date:	
Reviewed:		Date:	

Retired:

Date: _____

Kettering Health Network (KHN) Organization-Wide Policy

KHN adopts this policy for Kettering Medical Center, Sycamore Medical Center, Grandview Hospital and Medical Center/Southview Hospital, Greene Memorial Hospital, Inc., Soin Medical Center, Fort Hamilton Hospital, Troy Hospital, Kettering Physician Network, all hospital off-sites, and KHN Support Services.

Laboratory Services-KHN

Binax NOW Legionella Urinary Antigen

MB.040.010.00

Page 1 of 3

I. PRINCIPLE

The Binax NOW Legionella Urinary Antigen Test is an immunochromatographic membrane assay to detect *Legionella pneumophila* serogroup I soluble antigen in human urine from patients with symptoms of pneumonia. It is intended to aid in the presumptive diagnosis of Legionella infection caused by *Legionella pneumophila* serogroup 1.

Alere BinaxNOW® *Legionella* is an immunochromatographic membrane assay to detect *Legionella pneumophila* serogroup 1 soluble antigen in human urine. Rabbit anti- *Legionella pneumophila* serogroup 1 antibody, the patient line, is adsorbed onto nitrocellulose membrane. Control Line antibody, is adsorbed onto the same membrane as a second stripe. Both rabbit anti-*Legionella pneumophila* serogroup 1 antibodies and anti-species antibodies are conjugated to visualizing particles that are dried onto an inert fibrous support. The resulting conjugate pad and the striped membrane are combined to construct the test strip. This test strip and a well to hold the swab specimen are mounted on opposite sides of a hinged, book-shaped test card.

II. SPECIMEN COLLECTION, TRANSPORT, AND HANDLING

A. Urine specimens should be collected in a standard container. Store at room temperature (15-30°C) and assay within 24 hours of collection. Specimens may be stored at 2-8°C for up to 14 days. Boric acid may be used as a preservative. Allow all specimens to equilibrate to room temperature before testing.

III. MATERIALS

- A. Binax NOW Legionella Urinary Antigen Test
- B. Reagent A : Citrate/Phosphate with Tween 20 and Azide
- C. Positive Control Swab: heat inactivated Legionella pneumophila serogroup I positive urine dried onto a swab
- D. Negative Control Swab: heat inactivated normal human urine dried onto a swab.
- E. Swabs provided into the kit. DO NOT USE OTHER SWABS.
- F. Timer

IV. STORAGE AND STABILITY

A. The kit is stored at room temperature (2-30°C). It is stable until the expiration date marked on the outer packaging and containers. Do not use the kit after the expiration date.

V. QUALITY CONTROL

- A. The Alere BinaxNow Legionella Urinary Antigen card contains internal quality controls and external quality controls.
 - 1. **Internal Controls:** The internal quality controls are built-in to each test card and are performed with each test.
 - a. **Positive Control:** The pink-to-purple line at the Control position is an internal positive procedural control. If capillary flow has occurred, this line will always appear. The internal control does not assure functionality of the specific detection reagents. It only assures that the device is working properly.
 - b. **Negative Control:** The clearing of the background color in the window provides a negative background control. The background color in the window should be light pink to white within 15 minutes and should not interfere with the reading of the test.

Kettering Health Network (KHN) Organization-Wide Policy

KHN adopts this policy for Kettering Medical Center, Sycamore Medical Center, Grandview Hospital and Medical Center/Southview Hospital, Greene Memorial Hospital, Inc., Soin Medical Center, Fort Hamilton Hospital, Troy Hospital, Kettering Physician Network, all hospital off-sites, and KHN Support Services.

Laboratory Services-KHN

Binax NOW Legionella Urinary Antigen

MB.040.010.00

Page 2 of 3

- c. Documentation of the internal control functionality is provided under Results Entry in Epic Beaker. Following the entry of the test result in Beaker, document the internal control as “pass” or “fail”. If the internal control failed, the test is invalid and must be repeated.
1. **External Controls:** The external quality controls are provided in the test kit and must be run with each new lot or each new shipment.
- a. Procedure for BinaxNow Swab Controls:
- 1) Allow reagents and cards to equilibrate to room temperature (15-30°C) before testing. Remove card from the pouch just before use and lay the test card flat. There are two holes on the inner right panel of the card. Insert swab into the **BOTTOM** hole. Firmly push upwards so that the swab tip is fully visible in the top hole. **DO NOT REMOVE SWAB.**
 - 2) Hold Reagent A vial vertically, ½ to 1 inch above the card. Slowly add **six (6)** free falling drops of **Reagent A** to the **BOTTOM** hole. Immediately peel off adhesive liner from the right edge of the test card. Close and securely seal the card. Read result in window 15 minutes after closing the card. Results read beyond 15 minutes may be inaccurate. However, the positive control swab specimen line may be visible in less than 15 minutes.

Control	Expected Results	Interpretation
Kit Positive Control Swab	Positive	Pink-Purple lines at the control and sample positions
Kit negative Control Swab	Negative	Pink to purple line at the control position and no line at the sample position.

- 3) Do not use the kit or report patient results if positive and negative controls do not perform as expected. Contact the Microbiology Supervisor.

VI. PROCEDURE

- A. Open the test device just prior to use, lay flat, and perform assay. Equilibrate patient urine to room temperature before testing.
- B. Dip a swab (provided in the kit) into the urine specimen to be tested, completely immersing the swab head. If necessary, touch the swab to the side of the urine container to remove excess liquid.
- C. There are two holes in the inner right panel of the device. Insert the swab into the **BOTTOM** hole, and gently push upwards so that the swab tip is visible in the top hole.
- D. **DO NOT REMOVE THE SWAB.**
- E. Hold the Reagent A vial vertically, 1 to 1½ inches above the device and slowly add **2 drops of Reagent A** to the **BOTTOM** hole.
- F. Immediately peel off the adhesive liner from the right edge of the test device. Close and securely seal the device.
- G. Read the result in the window 15 minutes after closing the device. Results read beyond 15 minutes may be inaccurate. However, some positive patients may produce a visible specimen line in less than 15 minutes.

Kettering Health Network (KHN) Organization-Wide Policy

KHN adopts this policy for Kettering Medical Center, Sycamore Medical Center, Grandview Hospital and Medical Center/Southview Hospital, Greene Memorial Hospital, Inc., Soin Medical Center, Fort Hamilton Hospital, Troy Hospital, Kettering Physician Network, all hospital off-sites, and KHN Support Services.

Laboratory Services-KHN

Binax NOW Legionella Urinary Antigen

MB.040.010.00

Page 3 of 3

VII. INTERPRETATION

- A. **Positive:** Two pink-purple colored lines appear; one line is at the control position and one line is at the sample position. A positive test means that the antigen was detected. Specimens with low levels of the antigen may give a faint sample Line. **Any visible line is positive.**
- B. **Negative:** One pink-purple line is at the control position (top half of the window) and there is no line at the sample position. The control line means that the detection part of the test was done correctly, but no *L. pneumophila* serogroup 1 antigen was detected.
- C. **Invalid:** If no lines are seen, or if only the sample line is seen (no control line), then the test is invalid. Invalid tests must be repeated. If the problem persists, call Binax Technical Services at 1-800-323-3199 for further instruction. Additionally, contact the Clinical Microbiologist or Microbiology Supervisor.

VIII. REPORTING RESULTS

- A. **Resulting in Epic Beaker**
 - 1. Select result entry and enter the specimen number.
 - a. Verify patient information matches patient information on the test card.
 - 2. Click on "Edit"
 - 3. Enter the result of positive or negative in the value box for the Urine Legionella Screen.
 - 4. Enter the internal qc result as "pass" or "fail".
 - a. Note: Patient results should not be reported if the QC fails. The test should be repeated.
 - 5. Final verify the results.

IX. CRITICAL DETERMINANTS

- A. The Reagent A vial must be held VERTICALLY for correct dispense volume.
- B. The Reagent A vial must be held 1 - 1½ inches from the bottom hole to correctly dispense drops.
- C. The test device must remain sealed in its protective pouch until just prior to use.
- D. Do not mix components from different kit lots.
- E. Do not use swabs other than those provided in the kit.
- F. Alere BinaxNOW Legionella has been validated using urine specimens only. Other specimens (e.g. plasma, serum or other body fluids) that may contain Legionella antigen have not been evaluated. The test cannot be used on environmental specimens (i.e. potable water)
- G. This test will not detect infections caused by other *L. pneumophila* serogroups and by other Legionella species. A negative antigen result does not exclude infection with *L. pneumophila* serogroup 1.
- H. The diagnosis of Legionnaire's Disease cannot be based on clinical or radiologic evidence alone. There is no single satisfactory test for Legionnaire's Disease.
- I. Excretion of Legionella antigen in urine may vary depending on the individual patient. Antigen excretion may begin as early as 3 days after onset of symptoms and persist for up to one year afterwards.

X. REFERENCES

- A. BinaxNOW Legionella Antigen Card Product Technical Insert, Alere Scarborough, Inc., Rev. 2017/02

Kettering Health Network (KHN) Organization-Wide Policy

KHN adopts this policy for Kettering Medical Center, Sycamore Medical Center, Grandview Hospital and Medical Center/Southview Hospital, Greene Memorial Hospital, Inc., Soin Medical Center, Fort Hamilton Hospital, Troy Hospital, Kettering Physician Network, all hospital off-sites, and KHN Support Services.