

Blood Culture

Presented By:

Saraswathy

102171

Blood Culture

Outline:

1. What is a blood culture?
2. Why are blood culture important?
3. When should a blood culture be performed?
4. Type of blood culture bottles
5. What volume of blood should be collected?
6. How many blood culture sets should be collected?
7. How many days of incubation are recommended?
8. Type of Bactec system
9. Processing of blood culture positive
10. Conclusion

1. What is a blood culture?

A laboratory test in which blood, taken from the patient, is inoculated into bottles containing culture media to determine whether infection-causing microorganisms (bacteria or fungi) are present in the patient's bloodstream.

Blood cultures are intended to:

- Confirm the presence of microorganisms in the bloodstream
- Identify the microbial etiology of the bloodstream infection
- Help determine the source of infection (e.g., endocarditis)
- Provide an organism for susceptibility testing and optimization of antimicrobial therapy

2. Why are blood cultures important?

- Most widely used diagnostic tool for the detection of bacteremia and fungemia.
- It is the most important way to diagnose the etiology of bloodstream infections and sepsis and has major implications for the treatment of those patients.
- Its establishes or confirms that there is an infectious etiology for the patient's illness.
- Provides the etiologic agent for antimicrobial susceptibility testing, enabling
- optimization of antibiotic therapy.
- Sepsis is one of the most significant challenges in critical care, and early diagnosis is one of the most decisive factors in determining patient outcome.
- Early identification of pathogens in the blood can be a crucial step in assuring appropriate therapy, and beginning effective antibiotic therapy as early as possible can have a significant impact on the outcome of the disease.

Providing adequate antibiotic therapy within the first 24-48 hours leads to:

- Decreased infection-related mortality (20-30%)
- Earlier recovery and shorter length of hospital stay
- Less risk of adverse effects
- Reduced risk of antimicrobial resistance
- Cost reduction (length of stay, therapy, diagnostic testing)

3. When should a blood culture be performed?

1. Clinical symptoms in a patient which may lead to a suspicion of a bloodstream infection are:

- undetermined fever ($\geq 38^{\circ}\text{C}$) or hypothermia ($\leq 36^{\circ}\text{C}$)
- shock, chills, rigors
- severe local infections (meningitis, endocarditis, pneumonia, pyelonephritis, intra-abdominal suppuration...).
- abnormally raised heart rate
- low or raised blood pressure
- raised respiratory rate

2. Blood cultures should be collected:

- as soon as possible after the onset of clinical symptoms;
- ideally, prior to the administration of antimicrobial therapy.

4.Type of blood culture bottles:

- Mycobacterial blood culture bottle (BD BACTEC myco/F Lytic)
- BD BACTEC™ Plus Aerobic medium
- BD BACTEC™ Lytic Anaerobic medium
- BD BACTEC™ Peds Plus™ medium

5. What volume of blood should be collected?

- 1-5 ml of blood



Preferred Sample Type



aerobic and anaerobic set
8-10 ml blood volume

paediatric bottle
1-3 ml blood volume

6. How many blood culture sets should be collected?

- Since bacteria and fungi may not be constantly present in the bloodstream, the sensitivity of a single blood culture set is limited.
- However, to achieve a detection rate of >99% of bloodstream infections, as many as four blood culture sets may be needed.

7. How many days of incubation are recommended?

- The current recommendation, and standard incubation period, for routine blood cultures performed by continuous-monitoring blood systems is five days for Aerobic, anaerobic and peads bottles
- 30 days for Myco-lytic bottles.

8.Type of Bactec system in SJMC

- BD BACTEX FX



- BD BACTEC 9050



9. Processing blood culture positive:

- Once blood culture positive, unload from Bactec FX or 9050.
- Culture on Blood agar, Chocolate Agar, MacConkey and CDC
- Do gram stain also
- Inform Doctor immediately once gram stain is ready

- A positive blood culture is a critical result and must be reported as soon as available, due to the immediate impact on patient care decisions.
- When reports are delivered rapidly, studies have shown broadly improved outcomes and efficiencies in patient management.

- Once organism grows, perform identification and sensitivity testing using VITEK 2.
- To release results once confirmed ID and sensitivity.

10. Conclusion

- When processed correctly, blood cultures provide clinically relevant information that can help improve patient outcomes,
- decrease length of hospital stay and reduce use of antibiotics.