

Collection and Ordering:

1. If provider/client has opted in to utilize Guardian, the Guardian order will be reflexed when the Toxicology order is placed in Copia. (NOTE: There will be no order on a paper requisition for Guardian. The provider/office will opt in or out to the testing through the appropriate sales team member and the information will be stored in Copia. **If utilizing paper requisitions, urine is acceptable if the status of baseline is unknown. When orders are placed in processing, the correct orders will populate.)

Order as it appears in Copia Order Screen- First Collection

Order Choice	Diagnoses	Sample ID	Sample Type	Priority	Lab	Billing	ABN Status	Cancel
Urine Drug Screen	None selected	T.B.D.	- Default -	Routine	Harvest	Insurance Bill	Not Required	X
Helix Guardian	None selected	T.B.D.	- Default -	Routine	Harvest	Insurance Bill	Not Required	X
Helix Guardian Baseline	None selected	T.B.D.	- Default -	Routine	Harvest	Insurance Bill	Not Required	X
Validity Screen	None selected	T.B.D.	- Default -	Routine	Harvest	Insurance Bill	Not Required	X

Specimen/Collection Type:

First Collection (Guardian Baseline)

1. Preferred- Buccal Swab
2. Acceptable- Sterile Urine
3. Expectation is that the first submission would be complete with one buccal swab and one sterile urine.

Subsequent Collection (Guardian)

1. Sterile Urine

Order as it appears in Copia Order Screen- Subsequent Collection

Order Choice	Diagnoses	Sample ID	Sample Type	Priority	Lab	Billing	ABN Status	Cancel
Urine Drug Screen	None selected	22045000018	- Default -	Routine	Harvest	Insurance Bill	Not Required	X
Validity Screen	None selected	22045000018	- Default -	Routine	Harvest	Insurance Bill	Not Required	X
Helix Guardian	None selected	22045000019	- Default -	Routine	Harvest	Insurance Bill	Not Required	X

Collection Devices:

1. Buccal Swab- Puritan HydraFlock Sterile Flocked Collection Device



2. Stability- 24 hours (subject to change as volume and testing increases)
3. Keep swabs at Room Temperature after collection and during transi