

## INTELISWAB COLLECTION PROCEDURE



Blow your nose into a tissue. If assisting someone, instruct them to blow their nose. **DO NOT** use tissue to clear out nasal passage. Discard tissue and wash hands thoroughly. Dry hands before starting the collection.



Fig. 1

### SWAB BOTH NOSTRILS

ADULT: 15 TIMES **AND** ADULT: 15 TIMES

WHEN COLLECTING FROM A CHILD  
SEE ADDITIONAL INSTRUCTIONS



Fig. 2

**ADULTS:** Insert flat pad of the device inside the **nostril**. **Circle around the nostril 15 times** while maintaining contact with the **inside wall of the nostril**. **SWAB BOTH NOSTRILS** (Fig. 1 and Fig. 2). If you are conducting a test on a child 15-17 years old or an adult who requires assistance, proceed by swabbing the individual.

**CHILDREN (14 AND UNDER):** When collecting from a child under the age of 15, slowly **circle the swab in EACH nostril a minimum of 4 times** while gently pressing against the **inside of the nostril**. This should take about 15 seconds. (Fig. 1 and Fig. 2).

**If you DO NOT swab BOTH nostrils 15 times (adult) OR 4 times (child), you may get a false result.**

LABEL TEST DEVICE WITH SMALL LABEL

PLACE DEVICE BACK IN PACKAGING

PLACE IN BIOHAZARD BAG

SEND TO LAB