




VA U.S. Department of Veterans Affairs
 Infection Prevention
 5 Minute Educator
 Glucometer Cleaning



1ST STEP: CLEANING

All blood and other bodily fluids must be thoroughly cleaned from surfaces before disinfection by the **purple-top** PDI Super Sani germicidal wipe. Open, unfold and use first wipe to remove any soil.

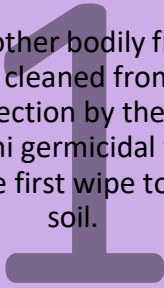

GLUCOMETER CLEANING

Per the instruction for use (IFU) of the Inform II Glucometer, **purple-top** PDI Super Sani-Cloths are to be used in cleaning and disinfection. Cleaning and disinfection is a two-step process done after every Veteran use.

Scan QR Code to watch video on Cleaning and Disinfection




2ND STEP: DISINFECTING

With a second **purple-top** PDI Super Sani germicidal wipe, thoroughly wet surface. Allow surface to remain wet for two minutes. Let air dry.

RESOURCES

[Joint Commission Focusing on Infection Control Risks: Glucose Monitoring and Insulin Administration](#)
[Glucometer Policy](#)

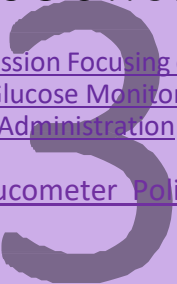

PLEASE NOTE

If working with a Veteran with potentially infectious diarrhea, follow the same 2-step cleaning process using the **Clorox Germicidal** wipes instead. This wipe has a 3-minute wet time.



