

ACCU-CHEK Inform II System Skills Checklist

Trainer should check activity as it is demonstrated or described.

User Assembles Equipment

- | | |
|---|---|
| <input type="checkbox"/> ACCU-CHEK Inform II system | <input type="checkbox"/> Blood sample collection materials –
(lancing device, gauze pad or cotton
balls, alcohol wipes (if used)) |
| <input type="checkbox"/> ACCU-CHEK Inform II test strips | |
| <input type="checkbox"/> ACCU-CHEK Inform II glucose control solution | |

User Performs Procedures

Coding (Calibrating ACCU-CHEK Inform II system)

- ☐ States when coding is necessary (when new lot introduced).
- ☐ Properly demonstrates the procedure for adding a test strip lot into the ACCU-CHEK Inform II meter (if user is required to do so).

Routine Testing Procedure

- ☐ Properly demonstrates the patient testing procedure including:
 - Proper infection control and safety practices (handwashing, donning personal protective equipment)
 - Assesses patient condition to ensure POC testing is appropriate.
 - Enters Operator ID according to policy
 - Enters Patient ID according to policy
 - Verifies test strip lot according to policy
 - Inserts test strip properly and recaps vial immediately
 - Performs fingerstick properly (stimulates blood flow, cleanses intended site, wipes first drop away, performs appropriate aftercare)
 - Applies blood to strip properly (fills window completely)
 - Reads and interprets result correctly
 - Enters comment(s) as needed according to policy
 - Understands proper follow-up to critically abnormal or unexpected results
 - Turns meter off
 - Discards testing materials according to policy
 - Documents result according to policy
 - Docks meter in base unit properly
 - Performs cleaning and disinfecting procedures as required by hospital infection control policy
- ☐ Properly demonstrates QC testing procedure including:
 - Understands when QC testing must be performed according to policy
 - Understands that controls expire 3 months from opening and ensures that the open expiration date is written on the control solution
 - Proper infection control and safety practices (handwashing, donning personal protective equipment)
 - Enters Operator ID according to policy
 - Enters control lot and level according to policy
 - Verifies test strip lot according to policy
 - Inserts test strip properly and recaps vial immediately
 - Applies control drop to strip properly (fills window completely)
 - Enters comment(s) as needed according to policy
 - Understands proper follow up to out-of-range QC results
 - Turns meter off
 - Discards testing materials according to policy
 - Documents result according to policy
 - Docks meter in base unit properly

Reviews Results

- Properly reviews results including:
- Enters operator ID according to policy
- Selects Review Results on the Main Menu screen to view list of most recent results
- Uses down arrow to view previous tests
- Uses up arrow to view most recent tests
- Selects Patient to view only patient results
- Enters patient ID according to policy to view specific patient results
- Selects All to return to viewing all results
- Selects QC to review only QC results
- Understands that the meter memory stores up to 2,000 results

Tips for Best Results and Troubleshooting

- ☐ Trainer reviews information from respective Operator's Manuals with the user:
 - Getting a good drop of blood
 - General hints
 - Proper test strip and test strip vial handling
 - Troubleshooting/Error codes
 - If problem persists, calls the ACCU-CHEK Customer Care Service Center at 1-800-440-3638

Cleaning and Disinfecting

- ☐ Demonstrates cleaning and disinfecting procedure of ACCU-CHEK Inform II system as stated in the Operator's Manual.
- ☐ Correctly states minimum cleaning and disinfecting frequency.

User Signature Date

Trainer Signature Date