



Daily Reagent QC: University and Methodist

QC Reagent Rack	Lot #/Expiration
COR QC Antisera	
COR QC Cells	
Monoclonal Ctrl	
Immucor Panoscreen	
Ortho Screen Cells	
Ortho Card	
All lot numbers were visually verified	Tech:
Appearance ✓ and Open Date/initials on all reagents	Tech:

Circle One: **University** **Methodist**

Date of Testing _____

Immucor Reagent	Reagent Rack
	Current Lot #/Expiration
Anti-A	
Anti-B	
Anti-D (series 4)	
A1 cells	
B cells	
LISS	
AHG	
Check Cells	
Saline	
All lot numbers were visually verified	Tech:
Appearance ✓ and Open Date/initials on all reagents	Tech:

Immucor Tube				Ortho Gel		Tech
Cell #	IS	37 LISS	AHG	OK?	IgG	
I				Y N		Y N
II				Y N		Y N
III				Y N		Y N

EXPECTED RESULTS	IS	37 LISS	AHG	Gel IgG
I/II	0	0-3+	1-3+	3-4+
III	0	0	0✓	0

Reagent		Anti-A	Anti-B	Anti-D	OK?	Tech
		A ₁				Y
B					N	
COR QC						

	Result	OK?	Tech
Waterbath # _____ (30-37°C)		Y N	
Bench Incubator # _____ (36-38°C)		Y N	
MTS Tach (1022-1042)		Y N	
MTS Timers (10/15/15)		Y N	
MTS Temp: Green light?		Y N	
Daily Bench and Equipment Decontamination Performed			
CW-2 fill (41-45 mL) (Monday)		Y N NA	

EXPECTED RESULTS	Anti-A	Anti-B	Anti-D
A ₁	3-4+		0
B		3-4+	
COR QC			3-4+

Scale QC	Result	OK?	Tech
20g		Y N	
50g		Y N	
100g		Y N	

Peer Review/Date: _____

Sup Review/Date: _____

Maintenance Reminder: Document on the applicable Form

MON—Eyewash Flush

WED— Clean Waterbath