TEMPERATURE HISTORY INDICATOR CARD

- CAUTION -

Before using RAD-SURE™ observe the color of attached film Temperature Indicator

IF BLUE/BLACK: Temperature history is good, RAD-SURE™ indicators are safe to use.

IF RED/ORANGE: Package overheated in transit, DO NOT USE RAD-SURE™ INDICATORS. From the US, contact Customer Service (855) 608-5639 to receive a replacement box of RAD-SURE™. From outside the US, contact your local distributor. For the name of a distributor in your area, contact the Product Manager in the US, tel: (908) 243-4766. fax: (908) 243-4773.

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Received
7/24/23
NM

temperature history indicator card

— caution —

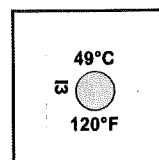
Before using this Ashland product, observe the change of the attached Temperature Indicator.

if white: Temperature History is within acceptable limit; this Ashland product can be used.

if black: Packaging overheated in transit; DO NOT USE this Ashland product. US customers can contact Customer Service to receive a replacement of this product by emailing DFOrdersNA@ashland.com

For the name of a distributor in your area, contact Customer Service by emailing DiagnosticFilmOrders@ashland.com

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© 2023, Ashland / PHA23-048_EN REV 00 / June 2023