



## PATIENT SETTINGS FOR THE SYSMEX *pochi100i*

### PURPOSE

To standardize the patient settings on the Sysmex *pochH-100i*

Patient limits setting on the Sysmex *pochH-100i* is as follows:

(Click on Menu / Settings / Patient Limits)

| Parameter | Lower | Upper | Parameter | Lower | Upper |
|-----------|-------|-------|-----------|-------|-------|
| WBC       | 3.5   | 11.0  | Mxd %     | 0     | 99.9  |
| RBC       | 2.00  | 7.00  | Neut %    | 0     | 99.9  |
| HGB       | 11.0  | 18.0  | Lym #     | 0.8   | 3.5   |
| HCT       | 20.0  | 60.0  | Mxd #     | 0     | 1.5   |
| MCV       | 70.0  | 110.0 | Neut #    | 2.0   | 10.0  |
| MCH       | 20.0  | 40.0  | RDW – SD  | 37.0  | 65.0  |
| MCHC      | 28.0  | 37.5  | RDW – CV  | 0     | 19.9  |
| PLT       | 150   | 450   | MPV       | 5.0   | 99    |
| Lym %     | 0     | 99.9  |           |       |       |