

How to Perform Patient Glucose Testing

Procedure (continued)



10. The text of Preprogrammed Comments is displayed in large buttons. Up to 3 preprogrammed comments can be selected. Press the appropriate comment(s) using your fingertip.



11. One custom comment can also be entered. From the Comment screen press the **Custom** button. Enter your comments by pressing the number and letter keys on the touchscreen using your fingertip. If you need to change your entry, press the ← key to backspace.

12. Press the large right-pointing arrow button under the touchscreen when finished entering comments to return to the Result Screen to view the result with the comments.



13. Press the large right-pointing arrow button or the **Menu** button under the touchscreen to complete the test and record the result. The display will return to the Main Menu screen.

Note: If the meter is powered off or allowed to time-out after a result is displayed, the result will be stored in the meter database.

Remove the test strip and discard according to your institution's infection control policy.