



**Dwight David Eisenhower Army Medical
Center Bldg. 300 East Hospital Road Fort
Gordon, Georgia 30905-5650**

**POINT OF CARE TESTING
BionaxNow Covid-19 Ag Card test**

NAME: _____ UNIT _____

EMPLOYEE # _____ DATE _____

COMPETENCY PERIOD:

____ INITAIL ____ 6 MONTH ____ ANNUAL SCORE _____
(OPERATOR MUST OBTAIN A SCORE OF AT LEAST 80%.)

1. Any shade of a pink horizontal line in the Test Result window is _____.

- a.) Positive
- b.) Negative
- c.) Questionable
- d.) Invalid

2. How many rotations counter clockwise should the swab be turned once it is inserted into the card?

- a.) 5
- b.) 7
- c.) 2
- d.) 3

3. The extraction reagent should be held vertically above the top hole of the testing card to dispense the reagent.

- a.) True
- b.) False

4. How many drops of the extraction reagent is needed to perform the test?

- a.) 5 drops
- b.) 3 drops
- c.) 6 drops

5. Can the test be read before the time required?

- a.) No
- b.) No, the test should be read after 15 minutes and no later than 30 minutes
- c.) Yes, can be read immediately after adding patient sample
- d.) Yes, any after the read time

6. External controls are to be ran

- a.) Once every 20 test
- b.) Once per each new shipment of kits
- c.) Every first sample tested each day
- d.) Once for each new lot and shipment of testing materials (kits and controls)

7. BinaxNow Covid-19 Ag test are ordered and resulted via OSO in CHCS?

- a.) True
- b.) False

8. What is the read time for the BionaxNow Covid-19 Ag test kit?

- a.) 15 minutes
- b.) 1 minute
- c.) 3 minutes

9. At what temperature should the BionaxNow kits need to be stored?

- a.) 2°C to 8°C (refrigerated)
- b.) 2°C to 30°C (refrigerated or room temperature)
- c.) -20°C (frozen)

10. False results can occur if?

- a.) 6 drops of the extraction reagent to the sample well
- b.) The swab is not rotated prior to closing the card
- c.) The swab is stored
- d.) all of the above