METHOD						
Methode // Méthode // Metodo // Método // Método // Metod // Metode						
			Level 1 - 15861		Level 2 - 15862	
			Mean	Range	Mean	Range
ERYTHROCYTE SEDIMENTATION RATE						
BD Seditainer (366065)	Glass	mm/hr	5	± 5	68	± 12
Diesse MiniVes	Plastic	mm/hr	10	± 10	80	± 18
Diesse MiniVes (Europe)	Plastic	mm/hr	▲		▲	
Diesse Ves-matic 10 (1)	Plastic	mm/hr	▲		▲	
Diesse Ves-matic 20 / 20 Plus New	Plastic	mm/hr	10	± 10	93	± 13
Diesse Ves-matic 20 / 20 Plus New (1)	Plastic	mm/hr	▲		▲	
Diesse Ves-matic 30 / 30 Plus (1)	Plastic	mm/hr	▲		▲	
Diesse Ves-matic 60 / 60 Plus (1)	Plastic	mm/hr	▲		▲	
Diesse Ves-matic Cube 30 (Citrates)	Plastic	mm/hr	▲		▲	
Diesse Ves-matic Cube 80 / 200 (Citrates)	Plastic	mm/hr	▲		▲	
Diesse Ves-matic Easy	Plastic	mm/hr	8	± 5	75	± 24
Diesse Ves-matic Easy (Europe)	Plastic	mm/hr	▲		▲	
Dispette2	Plastic	mm/hr	4	± 4	49	± 14
Greiner Bio-One SRS100, SRS20, SRT10 (1)	Plastic	mm/hr	§		§	
Kimble-Chase Disposable Westergren Tubes (2)	Glass	mm/hr	4	± 4	57	± 15
Polymedco Sedimat 60	Plastic	mm/hr	4	± 4	47	± 10
Polymedco Sediplast	Plastic	mm/hr	3	± 3	54	± 15
RR Mechatronics StaRRsed	Glass	mm/hr	▲		▲	
RR Mechatronics StaRRsed (1)	Glass	mm/hr	▲		▲	
Sarstedt S-Sedivette	Plastic	mm/hr	▲		▲	
Streck ESR-Auto Plus	Glass	mm/hr	9	± 7	69	± 18
Vital Diagnostics Excyte 20	Glass	mm/hr	8	± 6	78	± 12
Vital Diagnostics Excyte 40	Glass	mm/hr	6	± 6	83	± 12
Vital Diagnostics Excyte M / 10	Glass	mm/hr	6	± 6	76	± 12
Vital Diagnostics Excyte Mini	Glass	mm/hr	6	± 6	81	± 12
Vital Diagnostics MICROsed-System (1)	Glass	mm/hr	§		§	
Vital Diagnostics Monitor Series (1)	Glass	mm/hr	§		§	
Westergren (Modified)	Plastic	mm/hr	4	± 4	52	± 10
Wintrobe	Glass	mm/hr	▲		▲	

FOOTNOTES // Fussnoten // Notes de bas de page // Note a pie' pagina // Notas al pie de página // Notas de rodapé // Fotnoter // Fodnoter // Dipnotlar // 脚注

ENGLISH

- (1) TEMPERATURE CORRECTION: Means and Expected Ranges are corrected to 18°C in accordance with the Manley table.
- (2) METHOD VARIATION: Tubes do not contain Sodium Citrate (Diluent).
- ▲ Data not available at the time of printing. Please inquire.
- § The data required to establish the means and acceptable ranges for this assay were not obtained due to limited assignment participation. If your facility is interested in participating in the Value Assignment Program for this assay, please contact your local Bio-Rad Sales or Technical Services Group.
- ❖ INTERNATIONAL USE ONLY - The following section contains data for methods that are not available for diagnostic use in the United States.

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LOT 15861

EXP 5-9-2016

Liquichek™ Sedimentation Rate Control Levels 1 and 2

REF	514	Level 1	4 x 9 mL
	515	Level 2	4 x 9 mL
	514X	Bilevel MiniPak	2 x 9 mL

EXP 2016-05-09

LOT 15860

Level 1 15861
Level 2 15862**ENGLISH****INTENDED USE**

Liquichek Sedimentation Rate Control is intended for use as a quality control to monitor the precision of erythrocyte sedimentation rate (ESR) laboratory testing procedures.

SUMMARY AND PRINCIPLE

The use of quality control materials is indicated as an objective assessment of the precision of methods and techniques in use and is an integral part of good laboratory practices. Two levels of control are available to allow performance monitoring within the clinical range.

For customers in Germany: Quality control materials are required for assessment of laboratory performance as described in the "Guideline for Quality Assurance of Medical Laboratory Examinations following the German Medical Association" (Riil-BÄK regulation).

REAGENT

This product is composed of stabilized human red cells suspended in a buffered fluid and preservatives.

STORAGE AND STABILITY

This product will be stable until the expiration date when stored unopened at 18 to 30°C. Once opened, this product will be stable for 31 days when stored tightly capped at room temperature (18 to 30°C). Avoid prolonged exposure of opened vials to light. Vials should be kept tightly closed after use to avoid evaporation.

DO NOT FREEZE. DO NOT EXPOSE TO EXCESSIVE HEAT.

This product is shipped under ambient conditions.

PROCEDURE

This product should be treated and analyzed the same as patient specimens and run in accordance with the instructions accompanying the instrument, kit, or reagent being used.

1. This product must be prepared with a fresh tube each time.
2. Gently invert vial until packed cells have been resuspended. Continue mixing for 30 seconds. Avoid foaming. DO NOT VORTEX.
3. Follow the manufacturer's directions for filling sample tubes. The classical Westergren procedure (for example, Chase Disposable Westergren tubes) DOES NOT require prediluting of control material.
4. After each use, wipe any residual material from threads of vial and inside cap. Promptly replace the cap and store as described in the Storage and Stability Section.
5. Dispose of sample tubes; DO NOT re-use sample tubes.

Dispose of any discarded materials in accordance with the requirements of your local waste management authorities. In the event of damage to packaging, contact the local Bio-Rad Laboratories Sales Office or Bio-Rad Laboratories Technical Services.

LIMITATIONS

1. This product should not be used past the expiration date.
2. This product is not intended for use as a standard.
3. Atypical physical appearance may indicate instability or deterioration of this product. If the recovered values are not within the expected ranges, retest with a new bottle of this product. If the same result is obtained, contact the local Bio-Rad Laboratories Sales Office or Bio-Rad Laboratories Technical Services.
4. Discoloration of the product may be caused by overheating or freezing during shipping or storage, this may also indicate deterioration of this product.

ASSIGNMENT OF VALUES

The mean values printed in this insert were derived from replicate analyses and are specific for this lot of product. The tests listed were performed by using manufacturer supported reagents and a representative sampling of this lot of product. Individual laboratory means should fall within the corresponding acceptable range; however, laboratory means may vary from the listed values during the life of this product. Variations over time and between laboratories may be caused by differences in laboratory technique, instrumentation calibrating method and reagents. It is recommended that each laboratory establish its own means and acceptable ranges and use those provided only as guides.

Refer to www.qcnet.com for insert update information.

SPECIFIC PERFORMANCE CHARACTERISTICS

This product is a stabilized liquid product manufactured under rigid quality control standards. To obtain consistent vial-to-vial assay values, this product requires proper storage and handling as described.

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