

Maintenance - Cleaning

Clean the Exterior and Touch Screen

1. Turn the analyzer power OFF
2. Clean the exterior of the analyzer using a damp cloth and a mild detergent
3. Clean the touch screen, with a soft nonabrasive cloth dampened with a mild glass cleaner
 - Do not spray cleaner directly on the screen, or use bleach to clean the display



Remove and Clean the Push Bar

1. Turn the analyzer Off
2. Tilting the bar slightly upwards, pull the bar straight out
3. Clean the push bar with warm water and a mild detergent
4. Rinse thoroughly and dry with a paper towel or soft cloth
5. To reinstall the push bar refer to the *Maintenance: Reinstalling* tab in this document



Maintenance - Cleaning

Remove and Clean the Fixed Platform

1. Turn the analyzer Off
2. Remove the waste bin container and discard the used test strips into an appropriate container
3. Remove the fixed platform by pull the entire assembly towards you
4. Clean the platform with warm water and a mild detergent
 - Rinse thoroughly and dry with a paper towel or soft cloth
 - Allow the calibration bars to air dry
5. Inspect the calibration bars for scratches, marks, or discoloration
 - Discard and replace the platform with a new one if the bars can not be cleaned
6. To reinstall the platform refer to the *Maintenance: Reinstalling* tab in this document



Maintenance - Cleaning

Remove and Clean the Holddown Plate

1. Turn the analyzer Off
2. Press upwards on the tab at the back of the plate
3. Pull the other end from its retaining hole
4. Clean the holddown plate with warm water and a mild detergent
 - Wipe with a damp cloth
 - If the holddown plate is extremely dirty, soak in warm water and a mild detergent to loosen the dried residue
 - Rinse and dry with a paper towel or soft cloth
5. Use a cotton-tipped swab, wetted with plain water
6. To reinstall the holddown plate refer to the *Maintenance: Reinstalling* tab in this document



Remove and Clean the Moving Table

1. Turn the analyzer Off
2. Pull the moving table towards you
3. Clean the moving table with warm water and a mild detergent
 - Rinse thoroughly and dry with a paper towel or soft cloth
 - Use care when drying around the pins
6. To reinstall the moving table refer to the *Maintenance: Reinstalling* tab in this document

